REF **423290**

III **ConvaTec**

ESENTA™

Adhesive Remover Sting Free (150ml Aerosol)

en

Indications: For use on adhesives, dressings and appliances on skin.

DESCRIPTION OF DEVICE & CONTENTS: ESENTA™ Medical adhesive remover spray sting free (150ml)

INTENDED USE: Removal of adhesives, dressings and appliances from skin.

TARGET PATIENT POPULATION: People with adhesives, dressings and appliances on skin.

INTENDED USERS: Ostomates, Patients, Health Care Professionals and Caregivers.

INSTRUCTION FOR USE:

- Spray around the edge of the adhesive and wait a few seconds.
- Gently peel adhesive away from the skin using additional spray to ease removal if necessary.

INFORMATION:

- Dries in a few seconds. Sprays at any angle.
- If during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

CAUTION:

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.
- Sterility is guaranteed unless the pack is damaged. Do not use the device if packaging is damaged and dispose of the device according to local regulations.

WARNINGS: Use only as directed. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. Keep out of reach of children. **EXTREMELY FLAMMABLE**

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