ConvaCare®

en Adhesive Remover Wipe

de Hautreinigungstuch

nl Huidreinigungsdoekje

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Caution

Consult instructions for use

Intended Use: Atraumatic removal of medical adhesives, wound dressings, and appliances.

Indications for use: For use on skin around stomas, and skin around wounds at risk of skin breakdown.

PRECAUTIONS

FOR EXTERNAL USE ONLY. Avoid contact with the eyes. Do not use directly on wound, burn or stoma. Use only around wound, burn or stoma.

WARNING: Use only as directed. Keep out of reach of children. Do not use near open flame.

DIRECTIONS

- Gently lift a corner of the skin barrier, flange, tape, or hydrocolloid dressing.
- Wipe the adhesive remover towelette between the skin and the adhesive, gently peeling the adhesive back while continuing to wipe.
- Wash with mild soap. Rinse with water and pat dry.
Note: If adhesive remover spills on clothing, wet with water; regular washing should remove stain.

First Aid Treatment: If swallowed, do NOT induce vomiting. Call physician. In case of contact with eyes, flush with water.

DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

STORAGE CONDITIONS

Temperature limit
10°C 50°F 30°C 86°F

Note: Should irritation occur, consult a healthcare professional.

For more information on ConvaTec products, contact us at the address below.

© 2022 ConvaTec Inc.

® indicates a registered trademark of ConvaTec Inc.

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ConvaTec Limited
UNOMEDICAL A/S
ConvaTec International Services GmbH
DKSH Taiwan Ltd.
ConvaTec (Australia) Pty Ltd.