# Combihesive Natura®

High Output Drainable Pouch / Ausstreifbeutel  
extra groß / Poche Vidable de Grande Capacité /  
Sacchetto a fondo aperto di grande capacità /  
High Output Ileostomiezak / High output pose /  
High output pose / Höglflödespåse /  
High Output pussi / Tæmanlegur poki fyrir mikið  
frálag / Vrečka za odvajanje velike količine

Instructions for Use / Gebrauchsanweisung / Conseils  
d'utilisation / Istruzioni per l'uso / Gebruiksaanwijzing /  
Brugsanvisning / Bruksanvisning / Bruksanvisning /  
Käyttöohje / Notkunarleiðbeiningar / Navodila za uporabo

## (en) ENGLISH

For the management of a large volume of stoma effluent for individuals who require a high capacity pouch.

**Intended Use:** Management of stoma output.

**Indications for Use:** For use with a colostomy or ileostomy.

**Intended User:** Health Care Professionals, Caregivers and Ostomates.


### FLANGE APPLICATION AND REMOVAL

Apply and remove the flange and pouch as indicated in the skin barrier flange instructions for use. This product may remain in place while bathing, showering, or swimming. Pat dry with a towel after exposure to water. Make sure the tap is closed prior to application.

### FILTER REPLACEMENT

To eliminate and deodorize gas, the pouch incorporates a filter case with a water repellent filter inside. To replace the filter, remove the filter case cover, replace the old filter with a new filter (tan side facing out), and reseal the filter case.


### USING THE OUTLET TAP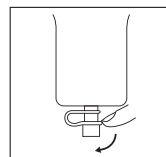


Remove the cap from the outlet tap and empty the pouch contents into the toilet or other approved container.

### USING THE TAIL CLOSURE

For drainage of more solid stoma output: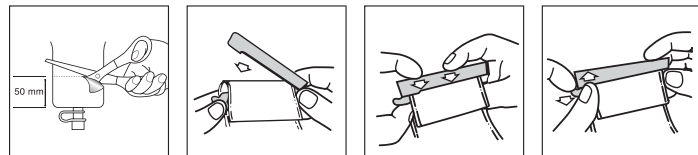

### USING THE DRAINAGE TAP

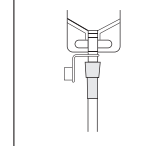Attach the connector provided to the outlet tap. Insert night drainage tubing into connector.

### DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

Note: Should irritation occur, consult a healthcare professional. For more information on ConvaTec products, contact us at the address below.

© 2021 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec (Australia) Pty Ltd.
DKSH Taiwan Ltd.
UNOMEDICAL A/S