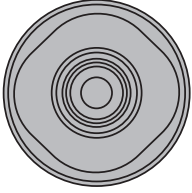
CE COLODRESS PLUS

en ONE PIECE STOMA CAP

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en INSTRUCTIONS FOR USE

tr KULLANIM TALİMATLARI

COLODRESS PLUS

en ONE PIECE STOMA CAP

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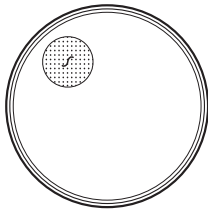
en The Colodress Plus Stoma Cap is designed for use by patients with a permanent end colostomy. It provides a cover for the stoma but has no capacity, and is therefore useful for wear over a short period of time or for people who irrigate and do not require the security of a closed pouch.

The Stoma Cap has an all-over skin barrier of Stomahesive, and is equipped with a filter for odour elimination or reduction and gas ventilation.

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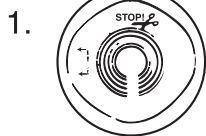
en The Stoma Cap may be cut to fit stomas up to a maximum of 50mm (2 inches).

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en The release paper has been printed with sizes 25mm, 32mm, 38mm, 45mm and 50mm to assist this process.

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en Using a small pair of scissors, preferably curved, enlarge the starter hole to match the stoma size.

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en Once the correct size has been cut out, remove the release paper and position the cap over the stoma pressing gently around the area to secure the adhesive.

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1

en Caps must **not be flushed** down the toilet as doing so may cause blockage of the sewage system.

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2

en Used caps may be disposed of through the refuse collection service. They should be wrapped in newspaper or in a plastic bag and sealed securely before being placed in the dustbin.

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3

en Some local authorities have a soiled dressings collection service. Details of this service can be obtained from the local council offices or your General Practitioner.

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4

en Used equipment must not be burned on a domestic fire or in an electric incinerator.

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en This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident

has occurred, please report it to the manufacturer and to your national authority.

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en All orders should be addressed to:
Sales Service Department, **ConvaTec Limited**, Unit 20, 1st Avenue,
Deeside Industrial Park, Deeside, CH5 2NU.

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globe Importer / İthalatçı

ConvaTec Inc.
Greensboro, NC 27409
USA
1-800-422-8811

ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd.
1800 339 412 Australia
0800 441 763 New Zealand

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Linus Medical UAB
Ukmergės g. 223,
LT-07156 Vilnius
Lithuania

www.convatec.com

© 2022 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.