ESTEM SYNERGY™

INSTRUCTIONS FOR USE (Must be used with the same size ESTEM SYNERGY™ pouch)

Intended Use: Management of stoma output.

Indications for Use: For use with a colostomy or ileostomy system.

PRECAUTIONS AND OBSERVATIONS: Patients with a flank hernia, when associated with megarectans and the absence of a urinary catheter, should know that there have been reported cases of adhesive bond in the urinary tract. Initiate appropriate clinical management and evaluate the clinical efficacy of the adhesive product if a mass is felt.

Important: A medical professional should assess the need for skin barriers with convexity prior to use.

1. Skin Preparation
The skin surrounding the stoma must be clean, dry, and free of any substances before applying any adhesive system.

2. Skin Barrier Preparation
A) Cut-To-Fit Skin Barrier (Figure 1)
Trace the shape of the stoma onto the skin barrier release liner. Do not cut beyond the outer tracing. Do not cut the skin barrier. Smooth cut opening with your finger.

3. Pre-Cut Skin Barrier
Select skin barrier with pre-cut stoma opening slightly larger than your stoma.

Notes: The flap overlay may cause skin irritation for users wearing digests. If irritation occurs, cut the cutaway layer on the skin barrier. Do not cut opening with your finger.

4. Applying the Patch
A) Skin barrier transparent landing zone. B) Push adhesive. (Figure 3)

Separate pouch flaps to allow air to enter. (Figure 4) Attach the matching pouch to skin barrier as follows: (Figure 5)

(A) Remove release liner from pouch adhesive ring. (B) Fold pouch adhesive ring in half without touching adhesive by grasping pull tabs without the adhesive ring fold shown. (C) Align bottom of pouch adhesive ring with bottom of landing zone and press in place.

Continue to apply with a bottom-to-top rolling motion. Run finger around pouch adhesive ring to ensure it is smooth and wrinkle-free. If difficulty is experienced, pouch can be removed and replaced. Refer to Section 5 on repositioning empty pouch.

5. Empty Pouch Can Be Repositioned
Apply Application (Figure 6)
Remove adhesive from landing zone by holding the bottom of pouch adhesive ring with one hand, and holding edge of the landing zone with the other hand, and gently peel downward with top-to-bottom rolling motion. Apply pouch as instructed in Section 4.

6. Closing The Pouch
A) Drainable Pouches with INVISIOCLE™ Tail Closure (Figure 7)
Fold tail closure beginning with the white outlet end. Repeat instructions for remaining pouches. Press interlocking closures together across the width of the tail and squeeze until you feel them snap. If desired, turn pocket inside-out to conceal the tail closure.

NOTE: For ESTEM SYNERGY Pouches do not have flap or cap.

Drainable Pouches with a Tail Clip (Figure 8)
Drainable Pouches with INVISIOCLE Tail Closure (Figure 9) Lock pouch closure and open the tail by pushing both ends of the outlet end-strips with fingers, drain pouch.

Drainable Pouches with Tail Clip (Figure 10)
Drainable Pouches with INVISIOCLE Tail Closure (Figure 11) Close the pouch by pulling the tail.

7. Emptying the Pouch
Drainable Pouches with INVISIOCLE Tail Closure (Figure 12)
Fold tail closure beginning with the white outlet end. Repeat instructions for remaining pouches. Press interlocking closures together across the width of the tail and squeeze until you feel them snap. If desired, turn pocket inside-out to conceal the tail closure.

NOTE: For ESTEM SYNERGY Pouches do not have flap or cap.

Drainable Pouches with a Tail Clip (Figure 8)
Drainable Pouches with INVISIOCLE Tail Closure (Figure 9) Lock pouch closure and open the tail by pushing both ends of the outlet end-strips with fingers, drain pouch.

Drainable Pouches with Tail Clip (Figure 10)
Drainable Pouches with INVISIOCLE Tail Closure (Figure 11) Close the pouch by pulling the tail.

8. Information About Pouches with Filters
Cover the filter with the sticker provided when bathing, swimming, etc. Remove after water activity.

9. Removing The Pouch
Change size on a do not forcefully remove the pouch. Use the following steps:

10. Removing The Skin Barrier (Figure 11)
1. Disposal
Place pouch in a plastic disposal receptacle. This device is for single-use only and should not be re-used. Do not use to fold increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with applicable medical waste and applicable local laws and regulations.

If during the use of this device or as a result of use, a serious incident has occurred, please report to the manufacturer and to your national authority.

© 2022 Convatec Inc. **™** indicates trademarks of Convatec Inc. ESTEM SYNERGY™ and INVISIOCLE™ are registered trademarks of Convatec Inc. in the United States.

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