ActiveLife®

One-Piece Drainable Pouch

Instructions for Use

III ConvaTec

For the management of stomal output.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

POUCH PREPARATION

The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

For pouches with a precut stoma opening: Select a pouch with the precut stoma opening at least 1/8 in. (3mm) **larger** than the largest stoma diameter.

DO NOT enlarge the precut stoma opening. Remove protective plastic shield from Stomahesive® disc.


For custom cut pouches:

Measure stoma with guide provided. Remove protective plastic shield from Stomahesive® disc.

Before cutting, separate front and back panels.


Using guidelines on release paper, cut the center hole the same size as the stoma. **Do not cut beyond last line.**

POUCH APPLICATION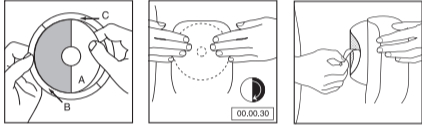

Remove release papers B and C. Ensure a snug fit around the stoma.

USING THE TAIL CLOSURE

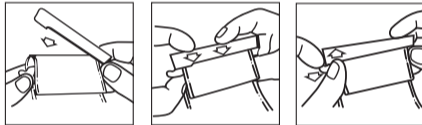POUCH REMOVAL

This product may remain in place while bathing, showering, or swimming. Pat dry after exposure to water.

DISPOSAL

Adhere to local disposal requirements/regulations.

STORAGE CONDITIONS

Temperature limit

Keep dry

Batch code

Use-by date

Read instructions for use.

Abbreviation for authorised representative in Switzerland

Importer

Note: Should irritation occur, discontinue use and consult a healthcare professional. For more information on ConvaTec products, contact us at the address below.

© 2025 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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®/™ são marcas da ConvaTec Inc.
ConvaTec Limited, First Avenue
ConvaTec (Australia) Pty Ltd.
DKSH Taiwan Ltd.