
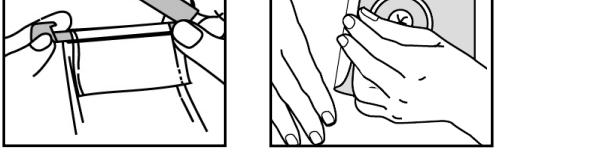
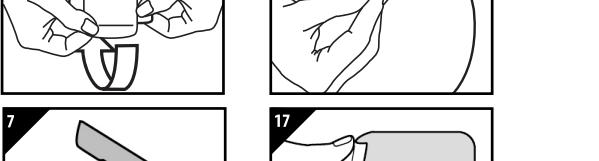
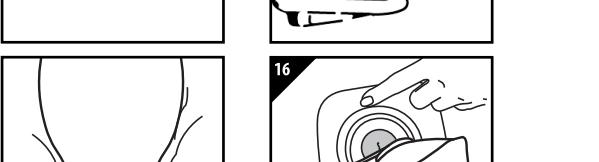
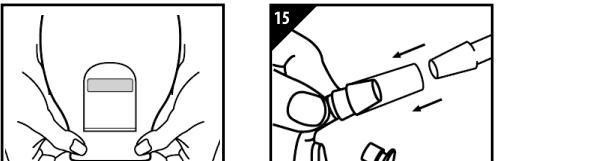
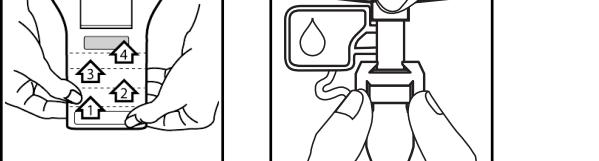
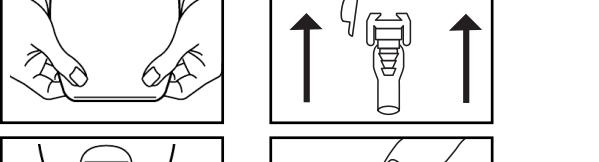
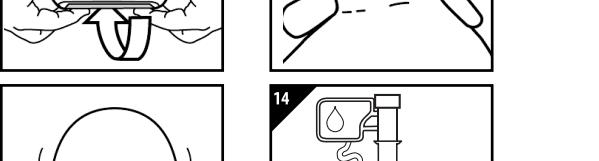
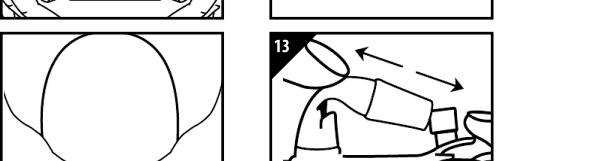
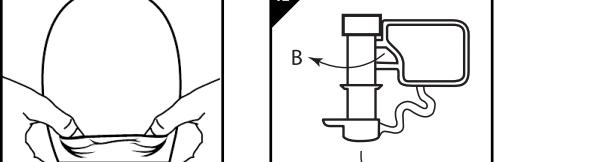
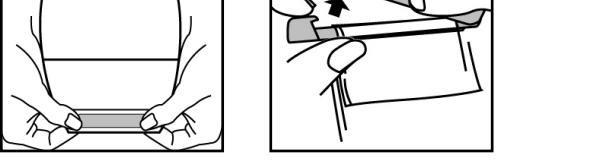
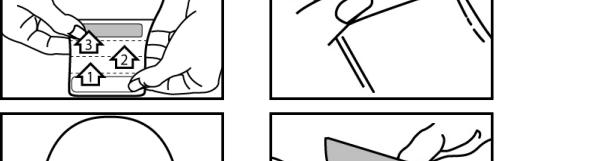
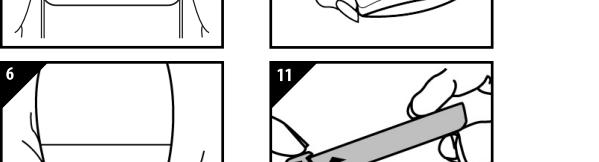
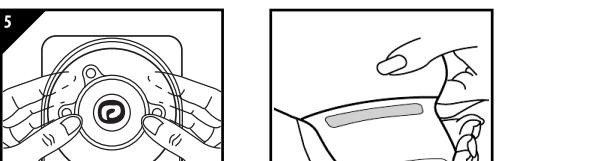
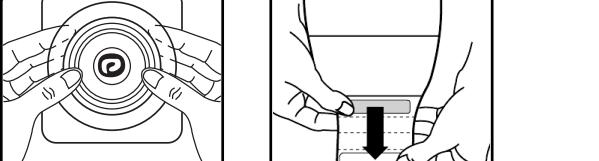
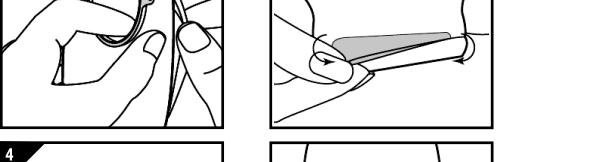
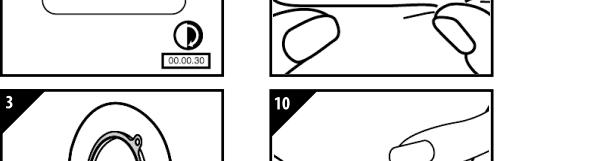
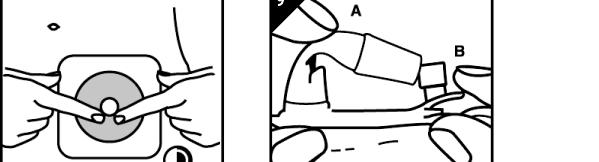
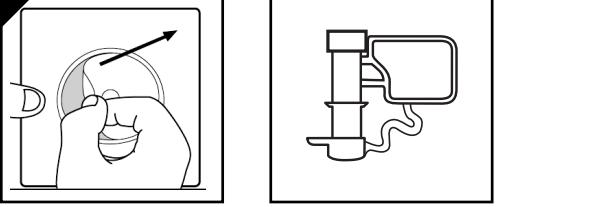


NATURA™

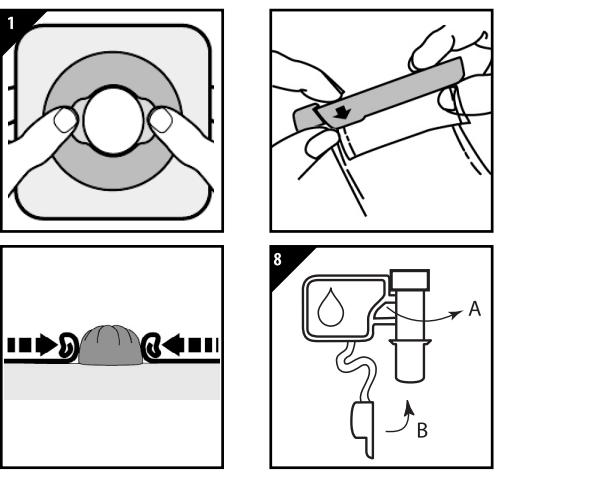

ENGLISH
Two-Piece Moldable Ostomy System -
Accordance Flange™

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
The Natura™ Moldable Skin Barrier - Accordion Flange™ allows the fingers to be placed between the skin barrier and the flange while coupling the coupling to the flange. It must be used with the same size Natura™, Sur-Fit™, and Combihesive™ Neutral Natura™ pouches.

PRECATIONS AND OBSERVATIONS
Caution: Patients with a flush ostomy, when associated with meatuses and the absence of a urinary reservoir, should know that there have been reported cases of adhesive found in the urine tract. Initial appropriate clinical evaluation and evaluate the continued use of the adhesive may be detected.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use.

Important: A healthcare professional should assess the need for skin barriers with prior to use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

INTENDED USE
Management of stomal output.

INDICATIONS FOR USE

For use with a colostomy, ileostomy or urostomy.

DIRECTS FOR USE

1. **SKIN PREPARE**

Wash hands and the area surrounding the stoma must be clean, dry and free from any substances before applying.

2. Skin Barrier Preparation (Figure 1)

Do not remove the clear border or release liner from the skin barrier. Skin barrier should be molded to stoma opening shape size without the use of scissors.

To prepare the stoma opening, place your thumbs (or fingers) at the edge of the stoma and begin to roll. Once the adhesive has been applied to the skin, the adhesive must be rolled. Do not stretch, pull or cut the adhesive to make it larger. Apply immediately following molding.

Notes: a) The tape collar can irritate skin irritation for users wearing diapers/nappies. If irritation occurs, cut the collar away from the skin barrier. b) Protect the stoma from trauma or injury.

3. Applying the Stoma System (Figure 2)

Remove all debris from the skin barrier and the coupling to the colostomy or ileostomy. Place the stoma into the center of the skin barrier. The adhesive must be present, remove remaining release liner. Smooth in place and wear without.

Press the rolled adhesive closest to the stoma down in a circular motion; hold in place for 30 seconds.

Wear as long as comfortable and secure. Wear time may vary.

4. Applying the Pouch

Separate pouches to allow air to enter. (Figure 3)

Hold the flap around three fingers of both hands. Line up the acordon flange and the pouch flange. (Figure 4)

If a small incision is present, remove the pouch by pulling the flap apart. (Pouch is available in Natura™ pouches only).

5. Closing the Pouch

A) Drawable Pouches with INVISCLOSE™ Fall Closure (Figure 6)

Fold tail closure beginning with the white outlet end-strip until interacting closure valves. Press interacting closure strips together until the flap is "Tucked" in place. (Pouch is available in Natura™ pouches only).

B) Snap Pouches with a Clip (Figure 7)

Closure of the pouch with ACCUSAL™ TAC (Figure 8)

D) Drawable Pouches with Fold over Top (Figure 9)

E) Snap Pouches with ACCUSAL™ TAC (Figure 10)

F) Drawable Pouches with a Clip (Figure 11)

G) Urostomy Pouches with ACCUSAL™ TAC (Figure 12)

H) Drawable Pouches with Fold over Top (Figure 13)

I) Snap Pouches with ACCUSAL™ TAC (Figure 14)

J) Connection to Night Drainage System: ACCUSAL™ cap (Figure 15)

Fold over the top. (Figure 16)

K) Interchangeable pouches with Filters (Figure 17)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 16)

10. Removing the Skin Barrier (Figure 17)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 18)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 19)

Fold over the top. (Figure 20)

14. Interchangeable pouches with Filters (Figure 21)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 22)

10. Removing the Skin Barrier (Figure 23)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 24)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 25)

Fold over the top. (Figure 26)

14. Interchangeable pouches with Filters (Figure 27)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 28)

10. Removing the Skin Barrier (Figure 29)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 30)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 31)

Fold over the top. (Figure 32)

14. Interchangeable pouches with Filters (Figure 33)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 34)

10. Removing the Skin Barrier (Figure 35)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 36)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 37)

Fold over the top. (Figure 38)

14. Interchangeable pouches with Filters (Figure 39)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 40)

10. Removing the Skin Barrier (Figure 41)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 42)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 43)

Fold over the top. (Figure 44)

14. Interchangeable pouches with Filters (Figure 45)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 46)

10. Removing the Skin Barrier (Figure 47)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 48)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 49)

Fold over the top. (Figure 50)

14. Interchangeable pouches with Filters (Figure 51)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 52)

10. Removing the Skin Barrier (Figure 53)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 54)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 55)

Fold over the top. (Figure 56)

14. Interchangeable pouches with Filters (Figure 57)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 58)

10. Removing the Skin Barrier (Figure 59)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 60)

Wipe off any residue from the pouch.

13. Connection to Night Drainage System: ACCUSAL™ cap (Figure 61)

Fold over the top. (Figure 62)

14. Interchangeable pouches with Filters (Figure 63)

Cover the filter with the filter provided when bathing, swimming, etc. Remove after water activity.

9. Removing the Pouch (Figure 64)

10. Removing the Skin Barrier (Figure 65)

11. Disposal of the product

After disposal, wash hands in accordance with accepted medical practice and applicable local laws.

12. Wiping off the pouch (Figure 66)

Wipe off any residue from the pouch.</p

