ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

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www.convatec.com

Revised 2022-08
1012745V2

(en) Flexible Tail Closure FOR DRAINABLE POUCHES

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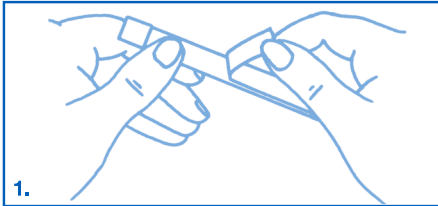
1. ENGLISH

Instructions for use

Intended Use: Closing drainable ostomy systems

Indications for Use: For use with a colostomy or ileostomy; and with compatible ConvaTec drainable ostomy systems.

IMPORTANT: Applying the tail closure as *described* is important to prevent leakage. Affix the tail closure *prior* to application of the pouch to the body.

- Remove release paper from adhesive strip. Position closure on the pouch side *without* the flange or skin barrier at the top of the finger notch, (approximately 1.2 to 1.5cm) from end of pouch tail. Press in place.
- Tightly fold pouch tail upward *4 times*.
- Tightly fold closure ends toward the pouch side with the flange or skin barrier; tail must be *smooth, flat, and not crimped*

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

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© 2022 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.