CE

0123

convatec

ConvaMax™

Superabsorber Adhesive Superabsorbant Adhésif

Superabsorber adhesive dressing / Superabsorber adhäsiv / Pansement superabsorbant adhésif / Medicazione Super-assorbente adesiva / Apósito superabsorbente adhesivo / Superabsorbčné adhezívne krytie / Υπεραπορροφητικό συγκολλητικό επίθεμα / Superabsorbierend adhesief verband / Pensó / Curativo adesivo superabsorvente / Opatrunek z superabsorbentem, przylepny

INSTRUCTIONS FOR USE / GEBRAUCHSANLEITUNG / MODE D'EMPLOI / ISTRUZIONI D'USO / INSTRUCCIONES DE USO / NÁVOD NA POUŽITIE / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / GEBRUIKSAANWIJZING / INSTRUÇÕES DE UTILIZAÇÃO / INSTRUKCJA UŻYCIA

en

ENGLISH

Description:

ConvaMax™ Superabsorber adhesive dressing with a silicone layer is a sterile dressing for use on highly exuding wounds, consisting of a breathable and waterproof nonwoven layer, a Superabsorber pad layer, a fabric wicking layer and soft silicone wound contact layer. The multi-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of faster wound closure. The absorbent pad provides superior absorption and locks exudates away. This minimizes the risk of maceration and damage to peri-wound skin. Super Absorbent Dressing absorbs and retains exudates and MMP's into the absorbent core to reduce the risk for maceration and encourage wound healing.

Indications:
ConvaMax™ Superabsorber adhesive dressing is indicated for management of moderate to highly exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds.

Directions for use:
Super absorbent dressing can be used by health care professionals and lay persons under supervision of health care professionals.
1. After the wound is cleansed, the appropriate size dressing is selected to allow for sufficient space to the wound edge.
2. ConvaMax™ Superabsorber adhesive dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, ConvaMax™ Superabsorber adhesive dressing must be in

close contact with the wound bed to ensure wound fluid is absorbed. White contact layer should be in contact with the skin and blue waterproof backing away from the body.
3. ConvaMax™ Superabsorber adhesive dressing is self-adhesive but may require additional support with a fixation dressing or mild-to-moderate compression bandaging.
4. The frequency of changing dressing depends on the level of exudate.
5. Dressing removal can be facilitated by moistening the dressing with saline solution.
6. Dispose of as clinical waste. If contaminated, follow local regulations

Contraindications:
1. Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.
2. ConvaMax™ Superabsorber adhesive dressing should not be used on patients with a known allergy or hypersensitivity to the dressing or its components.
3. Arterial bleeds and heavily bleeding wounds.

Caution:
1. Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.
2. Do not cut the dressing, the cover must remain intact.
3. In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

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© 2025 Convatec
®/™ označuje ochrannú známku spoločnosti Convatec Inc.