URODRESS

ONE PIECE UROSTOMY POUCH WITH STOMAHESIVE SKIN BARRIER

Product Description

Urodress is a one piece urostomy pouch specifically designed for people who need the protection of a Stomahesive skin barrier and prefer a one piece appliance. Urodress offers the following benefits:

1. An overall Stomahesive skin barrier giving maximum skin care.
2. The choice of pre-cut sizes (25mm – 45mm) for ease of application, or a 19mm starter hole for people who prefer to tailor the pouch to fit their own stoma.
3. A soft, quiet odour-proof film.
4. A simple fold-up tap for easy and secure closure.
5. The facility to fit a belt if required.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

Cleaning the Stoma

1. Clean the skin around the stoma with mild soap and warm water.
2. Rinse the area well and pat dry.
3. When using the Stomahesive products, it is best to avoid using barrier creams and solutions as they impair good adhesion.

Preparing the New Pouch for Application

Selecting the size of Urodress required

A. Pre-Cut Holes

1. Measure the maximum diameter of the stoma and select a hole size which is the same size or a little larger than the base of the stoma.

B. 19mm Starter Hole

1. Remove the outer plastic covering from Stomahesive skin barrier.

LEAVE THE RELEASE PAPER IN PLACE

2. Trace the outline of the stoma onto the release paper on the back of the Stomahesive skin barrier, using the guidelines to help you.
3. **IMPORTANT:** before cutting the skin barrier to size, separate the front and back panels of the pouch so as not to cut the front panel.
4. Using a small pair of scissors, preferably curved, enlarge the starter hole to match the traced outline of the stoma.

DO NOT CUT BEYOND THE LAST GUIDELINE CIRCLE

Before applying the Pouch

1. Check that there will be no areas of exposed skin between the edge of the hole and the base of the stoma. If this is likely to occur, or the skin around the stoma is uneven, depressed or creased, a filler such as Stomahesive paste may be used to protect the skin.
2. **ENSURE THE TAP IS CLOSED BEFORE APPLYING THE POUCH.**
The pouch is supplied with the tap open. To close the tap, bend the outlet spout (A) up to meet plug (B).
3. Press the plug firmly into the spout for easy and secure closure.

NOTE: The outlet spout is designed for the attachment of a night drainage bag if required. The Night Drainage System (Code No. S320) is available for this purpose.

If another night drainage system is used, the enclosed adaptor allows attachment of the drainage bag to the outlet spout.

Applying the Pouch

1. Remove the outer plastic covering from the Stomahesive skin barrier. (Those using a 19mm Starter will already have done this).
2. Remove the release paper from the skin barrier. (If using a 19mm Starter hole ensure the correct size has been cut out prior to removal of the release paper.)
3. Hold the pouch by the edges of the skin barrier and position over the stoma, making sure that no areas of skin are left exposed.
4. Press the Stomahesive barrier against the skin, paying particular attention to the area closest to the stoma. Maintain gentle pressure on the skin barrier to ensure good adhesion.

Urodress may be worn for as long as it is comfortable and there are no signs of irritation.

Urodress pouches need not be removed while bathing.

Removing the Used Pouch

1. Drain the pouch before removal.
2. Starting at the top, gently pull the adhesive barrier away from the skin with one hand while applying light pressure to the surrounding area with the other hand as the barrier is peeled downwards.
3. Small amounts of adhesive residue which may remain on the skin when changing the pouch need not be completely removed.

Disposal of Used Pouches

- 1 Pouches should be emptied before disposal. They must **not** be flushed down the toilet as doing so may cause blockage of the sewage system.
- 2 Used pouches may be disposed of through the refuse collection service. They should be emptied, rinsed out and wrapped in newspaper or in a plastic bag and sealed securely before being placed in the dustbin.
- 3 Some local authorities have a soiled dressings collection service. Details of this service can be obtained from the local council offices or your General Practitioner.
- 4 Used equipment must not be burned on a domestic fire or in an electric incinerator.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.


After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

URODRESS

ONE PIECE UROSTOMY POUCH WITH STOMAHESIVE SKIN BARRIER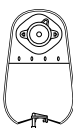

Ordering Information
Urodress One
Piece Urostomy Pouches

Product Code

19mm Starter hole	Clear	10 in Pack	S896
25mm Pre-cut hole	Clear	10 in Pack	S897
32mm Pre-cut hole	Clear	10 in Pack	S898
38mm Pre-cut hole	Clear	10 in Pack	S899
45mm Pre-cut hole	Clear	10 in Pack	S900
Accessory Products			
Orabase Paste		30g	S103
Orabase Paste		100g	S104
Stomahesive Paste		60g	S105
Orahesive Powder		25g	S106
Night Drainage Bag and 1 handle		5 in Pack	S320

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