Combihesive® Natura®

Stomahesive® Flange
Stomahesive® Flexible Flange
Stomahesive® Flange
with Flexible Collar

Instructions for Use

 ConvaTec

-  Do not re-use
-  Caution
-  Consult instructions for use
-  Medical Device
-  Importer

STORAGE CONDITIONS

 10°C / 50°F to 25°C / 77°F Temperature limit

 Keep dry

 Batch code

Intended use: Management of stomal output.

Indication for Use: For use with a colostomy, ileostomy or urostomy.

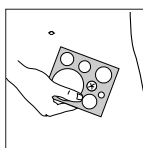
PRECAUTIONS AND OBSERVATIONS


CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.


Note: Should irritation occur, consult a healthcare professional.

FLANGE PREPARATION


The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

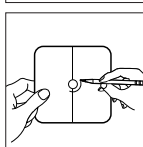 Select a flange size that is 6 - 10 mm **larger** than the widest diameter of the stoma.

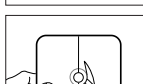 Using the cutting guide, trace the exact stoma outline onto flange release paper.


 Cut along tracing leaving at least 6 mm of skin barrier between the enlarged starter hole and the flange ring. Remove release paper.

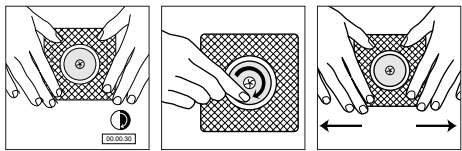FLANGE APPLICATION


Ensure a snug fit around the stoma.

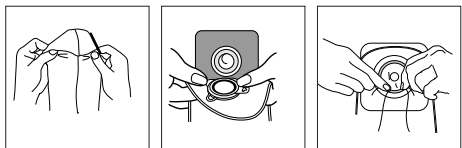POUCH APPLICATION


This product may remain in place while bathing, showering, or swimming. For closed pouches with a filter, cover filter with filter covers provided. Pat dry after exposure to water.

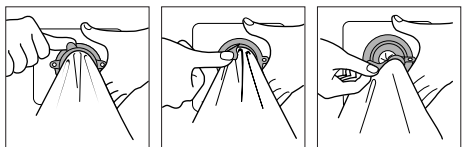POUCH REMOVAL


FLANGE REMOVAL

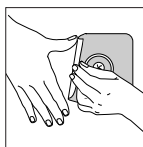USING THE INVICLOSE™ OUTLET DIRECTIONS TO CLOSE THE POUCH:

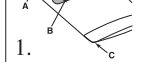
The InvisiClose tail closure system (1) is comprised of a security flap (A), inter-locking closures (B) and outlet end-strips (C).

Hold the security flap against the front of the pouch revealing the inter-locking closures. (2)

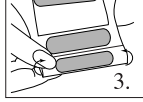Fold the tail end up towards you four times and ensure the inter-locking closures overlap. (3)

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

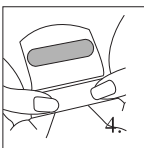
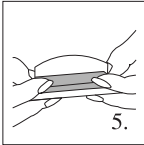
Press the inter-locking closures together until you feel them lock (CLICK) into place along their entire length. (4)

Wrap the security flap around to the back of the pouch (body side) and press the inter-locking closures together until you feel them lock (CLICK) into place. (5)

FOR DRAINING THE POUCH:

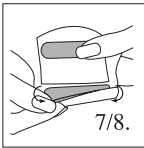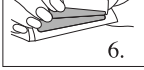Turn the back of the pouch (body side) towards you. Carefully lift the security flap up towards you to unlock. (6)


While holding the security flap up against the front of the pouch, unlock the inter-locking closures by lifting at the corner (7) and unraveling the folds four times. (8)


To facilitate draining and/or cleaning, you may open the tail end by pushing in both ends of the outlet end-strips with your thumb and forefinger. (9)

If desired, pouch tail may be cleaned with gentle soap and water. Pat dry.

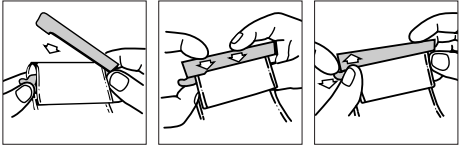
USING THE TAIL CLOSURE


USING THE ACCUSEAL® TAP

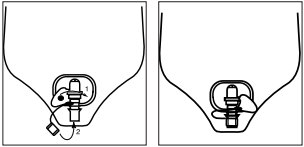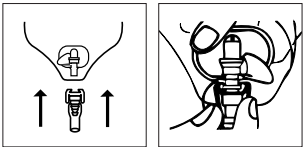
For connection to a night drainage system:

USING THE FOLD OVER DRAINAGE TAP

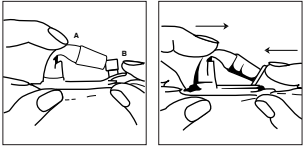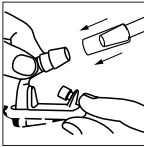For connection to a night drainage system:

DISPOSAL

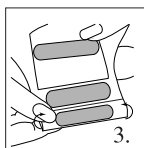
This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

For more information on ConvaTec products, contact us at the address below.

© 2021 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Limited
UNOMEDICAL A/S