INSTRUCTIONS FOR USE

Intended Use: Management of stomal output.

Indications for Use: For use with a colostomy or ileostomy.

- Skin Preparation**
Your hands and the skin surrounding the stoma must be clean, dry and free from oily substances before applying.
- Pouch Preparation (Figure 1)**
Trace the shape of the stoma onto the skin barrier release liner. Cut the skin barrier just outside the tracing taking care not to cut pouch film. Do not cut beyond last line. Smooth cut opening with your finger.
Note: Protect the stoma from trauma or injury.
- Applying the Pouch (Figure 2)**
Separate pouch films to allow air to enter. Remove release liner. Center the opening of the skin barrier over the stoma. Apply skin barrier against skin around stoma for 30 seconds.

Wear as long as comfortable and secure. Wear time may vary.

- Closing the Pouch (Figure 3)**
Fold tail closure beginning with the white outlet end-strip until interlocking closures overlap. Press interlocking closures together across the width of the tail and squeeze until you feel them close. If desired, turn pocket inside-out to conceal the tail closure, or wrap flap down around pouch tail.
- Emptying the Pouch (Figure 4)**
Unlock pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers; drain pouch.

- Removing the Pouch (Figure 5)**
- Disposal**
This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

© 2022 ConvaTec Inc.
*/™ indicates trademarks of ConvaTec Inc. ESTEEM is a registered trademark of ConvaTec Inc. in the United States.

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*/™ ist ein eingetragenes Warenzeichen der ConvaTec Inc.