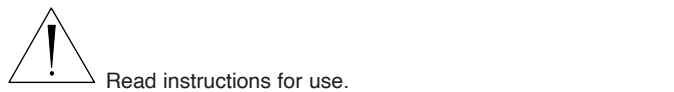
Visi-Flow®

Irrigation System

Irrigator with Stoma Cone

Instructions for Use

Compatible for use with both the SUR-FIT® and SUR-FIT® Natura™ Ostomy Systems.

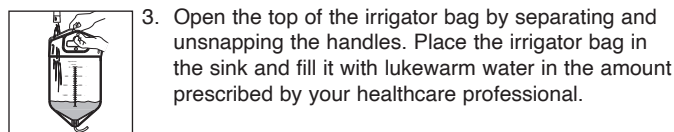
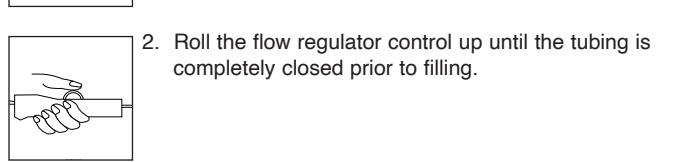
PRODUCT DESCRIPTION
For persons with colostomies for whom irrigation is indicated for evacuation of stool from the colon.

- WARNINGS AND PRECAUTIONS**
- This irrigation method may not be suitable for all stoma types. Check with your health care professional before you begin the irrigation process to determine the best irrigation method for your stoma.
 - Ensure the stoma cone is clean and undamaged before insertion to help prevent injury to the stoma.
 - Consult a healthcare professional if persistent bleeding from the stoma is experienced during or after irrigation.
 - Persons receiving pelvic or abdominal radiation should consult with their healthcare professional prior to irrigation.


CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

PREPARING AND FILLING THE IRRIGATOR BAG



- Attach stoma cone to tubing. Note: If your healthcare professional has recommended the catheter method of irrigation, attach the appropriate catheter instead of the stoma cone.




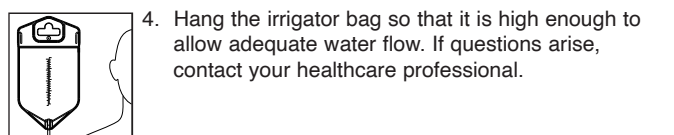
Check the water level in the irrigator bag by holding it in an upright position with the top of the bag closed. Once the water level is correct, snap handles closed.

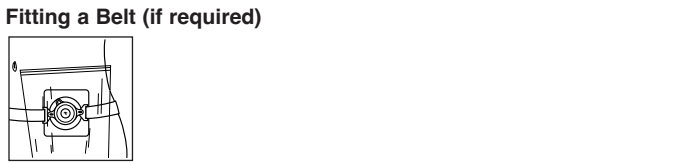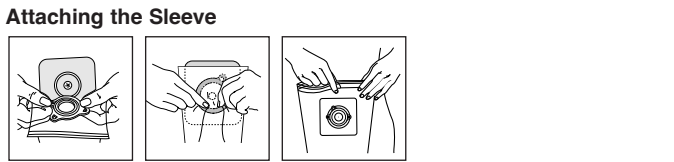USING THE IRRIGATOR WITH THE IRRIGATION SLEEVE
Apply wafer as directed in instructions for use provided with wafer at least one hour prior to beginning the irrigation procedure.

- INSTRUCTIONS FOR THE IRRIGATION PROCESS**
IMPORTANT: Before starting the irrigation process:
- Sit on the toilet or on a chair next to the toilet.
 - Position the bottom of the irrigation sleeve so it hangs into the toilet bowl.
 - Remove entrapped air from the tubing by opening the flow regulator to allow a small amount of water to run through the stoma cone into the toilet.
 - Close the flow regulator.

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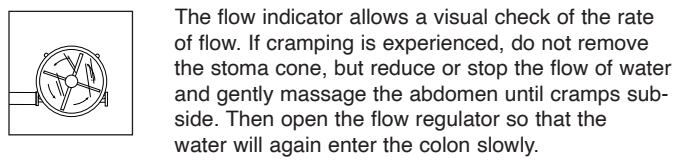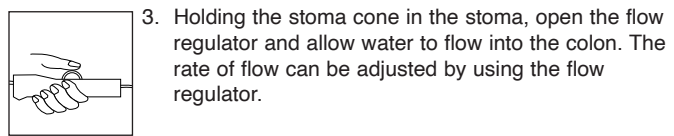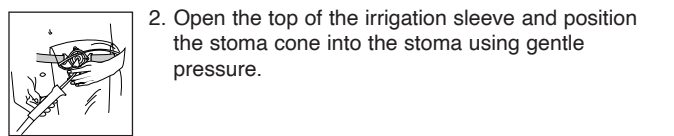
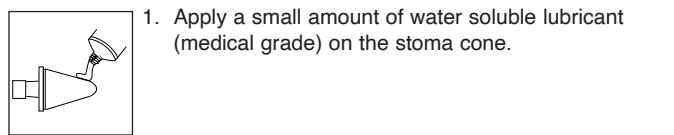
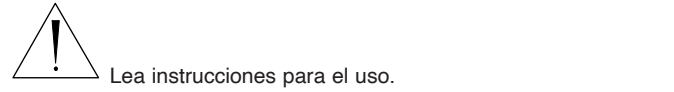
4. After the prescribed amount of water has entered the colon, stop the water flow by rolling the flow regulator closed. Remove the cone from the stoma and seal the top of the sleeve using the slide seal. Use brush supplied with starter kit to clean stoma cone.

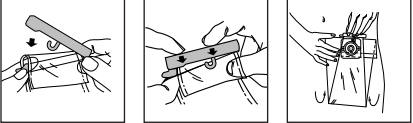5. After initial evacuation of stool, attach the tail closure to the tapered bottom of the irrigation sleeve and hook to release tab of sleeve.

To open tail closure:

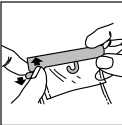6. After evacuation is complete, rinse the sleeve by running water through the top opening.

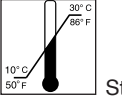
7. Remove the irrigation sleeve from the wafer by pulling the tab located at the top of the sleeve flange. Maintain gentle pressure on the wafer as the sleeve is removed.

8. Make sure the wafer is clean and dry before applying a pouch.

DISPOSAL
Dispose of used irrigation system components in adherence with local disposal regulations/requirements. Do **not** flush components down the toilet.

STORAGE CONDITIONS

Storage temperature.

Keep dry. Avoid high humidity.

Lot/control number

Expiry date

Note: Should irritation occur, discontinue use and consult a healthcare professional.

For more information on ConvaTec products, contact us at the address below.

© 2024 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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