Enhanced Hydrofibrin™ Dressing with Silver and Strengthening Fibre / Verbesterde Aquazol™ Extra is a soft, sterile, non-woven dressing made from two layers of silver carboxymethylcellulose impregnated with 1.2% silver ions (antimicrobial biocide), ethylene dimethylammonium chloride (EDTA) and benzethonium chloride (BEC), and strengthened by regenerated cellulose cellulose.

The dressing absorbs moisture of wound fluid and bacteria and creates a soft, cohesive gel that intimately conforms to the wound surface, maintains a moist environment and aids in the removal of exudate. The fine woven structure of the dressing provides a barrier to infection, environment and control of wound bacteria supports the body's healing process and helps reduce the risk of wound infection. The soft, silver in the dressing kills pathogenic micro-organisms and reduces the risk of infection. The dressing also disrupts and absorbs biofilm, which has been shown to reduce bacterial adherence to the wound surface and prevent biofilm microorganisms.

The dressing itself also provides an antimicrobial barrier to protect the wound bed.

INTENDED USE
Aquazol™ Extra Dressings have been designed to be used as a primary dressing. They are intended to be used under the direction of a healthcare professional for wounds, which are at risk of infection or show signs of infection, or where biofilm is suspected to be present, and in combination with the following medical treatments:

INTENDED USERS
Aquazol™ Extra Dressings are intended to be used by healthcare professionals, carers and patients under the direction of a Healthcare Professional.

TARGET PATIENT POPULATION
Aquazol™ Extra Dressings are designed to be used on patients with one of the wound types listed in the indications for use.

CLINICAL BENEFITS
Aquazol™ Extra Dressings absorb wound fluid, bacteria and biofilm, providing a moist healing environment, aiding autolytic debridement and removing dead space between the wound and dressing interface.

Aquazol™ Extra Dressings are designed to manage excess exudate levels which may further damage the wound bed and surrounding skin.

Aquazol™ Extra Dressings provide a barrier to protect the wound bed from contamination.

Aquazol™ Extra Dressings effectively kill bacteria, yeasts and moulds in both planktonic and biofilm form.

INDICATIONS
Aquazol™ Extra Dressings are indicated for:

- Acute or chronic ulcers
- Arterial ulcers
- Leg ulcers of mixed aetiology
- Diabetic foot ulcers
- Pressure ulcers/stages
- Surgical wounds
- Traumatic wounds
- Ulcers of the lower limb
- Partial thickness burns

CONTRAINDICATIONS
Aquazol™ Extra Dressings should not be used on individuals who are sensitive to or who have had an allergic reaction to silver, sodium carboxymethylcellulose, ethylene dimethylammonium chloride or sodium ion. Allergic reactions to benzethonium chloride.

PRECAUTIONS AND OBSERVATIONS
This dressing is not intended for use as a surgical sponge.

Aquazol™ Extra Dressings are for single use only and should not be re-used. Re-use may lead to increased risk of infection, cross contamination and delayed healing.

Aquazol™ Extra Dressings are not compatible with petroleum-based products. Use of the sterilisation process there may be a slight odour on opening the primary packaging.

During the body's normal healing process, non-viable tissue is removed from the wound (autolytic debridement), which could initially make the wound appear larger.

It is important to monitor the dressing for signs of infection, increased pain, bleeding, warmth, redness of surrounding tissue, wound enlargement, or a change in colour or odour.

After use, this product may be a potential biohazard and should be disposed of in accordance with accepted medical waste disposal and regulatory requirements.

If during the use of this device or at any point of use a serious incident has occurred, please contact the manufacturer or your local regulatory authority or the Member State in which you and/or patient is established.

USER AND PATIENT CARE INSTRUCTIONS
Aquazol™ Extra Dressings should be used alone or in combination with other wound care products such as primary dressings, secondary dressings, occlusive bandages, compression therapy devices or medical topical treatments. Use of additional products and therapies should be in accordance with the directions of the manufacturer.

CONTACT AND DURATION OF USE
Aquazol™ Extra Dressings can be worn for up to 7 days, dressings should be changed daily.

On partial thickness (second degree) burns, dressings may remain in place for up to 14 days. The requirement for changing Aquazol™ Extra Dressings should be reassessed after 14 days and alternative wound management considered where appropriate.

DIRECTIONS FOR USE
Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.

Aquazol™ Extra Dressings should overlap at least 1 cm on the skin surrounding the wound.

When using Aquazol™ Extra Dressings in deep wounds, only fill the wound up to 80% as Aquazol™ Extra Dressings will expand to fill the wound space on contact with wound fluid.

A secondary dressing is required to hold Aquazol™ Extra Dressings in place, such as a moisture retentive cover dressing such as DuoDERM™ Extra™ Thin in lightly to moderately moist wounds.

A secondary cover dressing such as AQUAFIX™ Foam or Convamax™, is moderate to heavily exuding wounds.

For 40% wounds refer to section FOR DRY WOUNDS below.

See individual cover dressing package inserts for complete instruction for use.

Aquazol™ Extra Dressings should be used under the direction of a Healthcare Professional when clinically indicated (i.e., leakage, exsiccated, increased pain) or after a maximum of seven days.

FOR PARTIAL THICKNESS SECOND DEGREE BURNS
Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.

Aquazol™ Extra Dressings should overlap at least 5 cm on the skin surrounding the wound and should adhere to the skin.

Aquazol™ Extra Dressings should be covered with a sterile absorbent pad and secured with medical tape or a retention bandage.

Remove the cover dressing and inspect the Aquazol™ Extra Dressings while the patient remains in place on the burn.

In this indication address to the wound bed of the Aquazol™ Extra Dressings is a desired characteristic.

As the burn wound re-epithelialises, the Aquazol™ Extra Dressings will detach or be easily removed.

As the burn wound progresses (second degree burn), Aquazol™ Extra Dressings may be left in place for up to 14 days or until clinically indicated. If the burn is infected, frequent changes of the wound may be necessary.

FOR DRY WOUNDS
Pre-moisten the Aquazol™ Extra Dressings by wetting with sterile saline prior to dressing the wound. The gelling properties of Aquazol™ Extra Dressings will help to draw the most wound exudate into the dressing.

Cover the dressing with a moisture retentive dressing such as DuoDERM™ Extra™ Thin to avoid drying out of the dressing and supporting addressing additional to the wound.

Remove the dressing if dry, hydrate with sterile saline prior to removal to reduce risk of trauma.

Store at room temperature (10°C - 25°C/50°F - 77°F). Keep dry.

For further information on Aquazol™ Extra Dressings contact Convatec Medical and Clinical Performance (SCCP) location after the launch of Enhanced Hydrofibrin (EDH) products (EU/UMDAEM).

For further information if guidance is needed, please contact Convatec Professional Services.

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© 2023 Convatec Inc.