Combihesive Natura®

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(en) ENGLISH

INTENDED USE


Management of stomal output for short periods of
time or following irrigation.

INDICATIONS FOR USE

For use with a colostomy and with compatible
ConvaTec ostomy wafers.

APPLICATION


Apply a Combihesive Natura® flange (see product
instructions for use).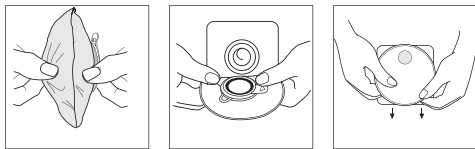

Filter must be at top of cap.

This product may remain in place while bathing,
showering, or swimming. Cover filter with filter covers
provided. Pat dry with a towel after exposure to water.

REMOVAL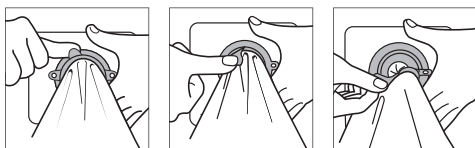


DISPOSAL

This device is for single-use only and should not be
re-used. Re-use may lead to increased risk of infection or
cross contamination. Physical properties of the device
may no longer be optimal for intended use.

If during the use of this device or as a result of its use a
serious incident has occurred, please report it to the
manufacturer and to your national authority.

After use, this product may be a potential biohazard.
Handle and dispose of in accordance with accepted
medical practice and applicable local laws and regulations.

STORAGE CONDITIONS

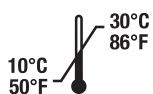MD Medical Device

Note: Should irritation occur, consult a healthcare
professional.

For more information on ConvaTec products, contact us
at the address below.

© 2022 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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