ActiveLife®

Urostomy Pouch with Durahesive® Skin Barrier and Accuseal® Tap

Instructions for use

ENGLISH - Instructions for Use

PRODUCT DESCRIPTION

The Active Life® Urostomy Pouch with Durahesive® Skin Barrier and Accuseal® Tap is a one-piece urostomy pouch with an adhesive support collar. The pouch is constructed with a nonreturn valve and ACCUSEAL tap.

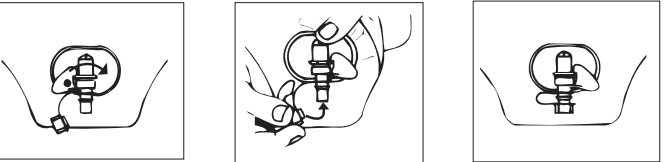
CONTRAINDICATION

Use on individuals with a known sensitivity to the adhesives, pouch, or the product components.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

POUCH PREPARATION AND APPLICATION

1. Cleanse the peristomal area with water and pat **thoroughly** dry, making sure it is free from any greasy substances or solvents.
2. **IMPORTANT: Select an ACTIVE LIFE Urostomy Pouch with DURAHESIVE Wafer and ACCUSEAL Tap with the precut stoma opening the same size as the largest diameter of the stoma. Do not enlarge the precut opening.**

Make sure the tap is in the closed position and the plug is in the tap stem.

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

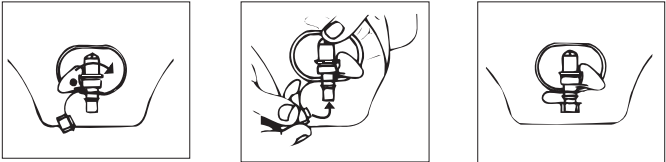
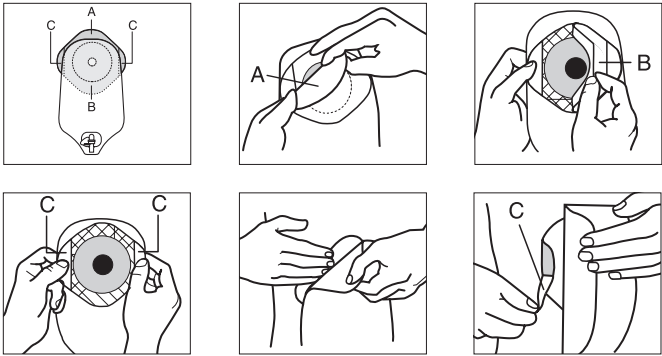
When Stomahesive® Paste is indicated, apply a ring of paste to the back (adhesive side) of the DURAHESIVE disc, around the precut opening. Allow the paste to set for one minute prior to application to the skin. Press the DURAHESIVE disc against the skin paying particular attention to the area closest to the stoma.

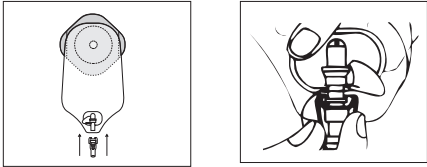USE OF THE ACCUSEAL ADAPTER


To attach to a night drainage container, use the adapter provided.

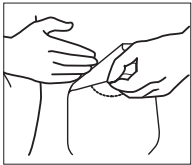POUCH REMOVAL

A water-soluble adhesive remover is recommended if the pouch is difficult to remove.

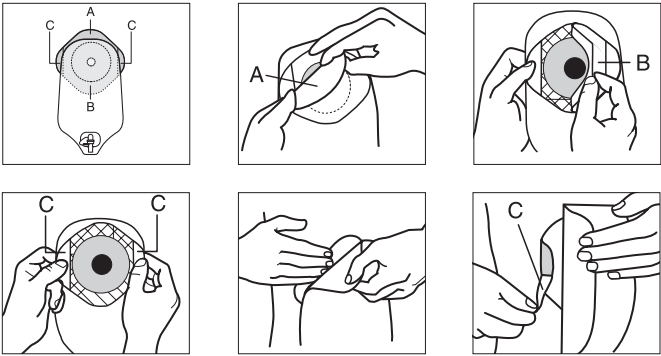®/™ indicates a trademark of ConvaTec Inc.

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© 2024 ConvaTec Inc.
MD/MC est une marque déposée de ConvaTec Inc.
ConvaTec Limited, Deeside, CH5 2NU, UK
ConvaTec (Australia) Pty Ltd.