ILEODRESS PLUS™

(en) ENGLISH

Drainable Pouches

with Stomahesive®
Skin Barrier

Instructions for use

Ileodress Plus™ Drainable Pouches are one-piece appliances consisting of an adhesive disc of Stomahesive™. Specifically designed for people who need the protection of the Stomahesive™ skin barrier but who prefer a one-piece appliance, they offer the following advantages:

- 1 The unrivalled protection of the Stomahesive™ Skin Barrier.
- 2 The specifically textured barrier on the back panel of the pouch prevents moisture accumulating thus increasing the comfort of the pouch where it touches the skin.
- 3 A soft, quiet odour-proof barrier film.
- 4 Pre-cut hole sizes for ease of application, or a starter hole (19 mm) for those people who prefer to tailor the appliance to fit the stoma.
- 5 The choice of clear or opaque pouches.
- 6 The choice of standard or small pouches.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.


General Instructions

- 1 Clean the skin around the stoma with soap and warm water and pat dry.
- 2 When using **Stomahesive Products**, barrier creams and solutions are best avoided as they prevent good adhesion.
- 3 If the surface of the skin around the stoma is uneven due to scarring, etc. such depressions may be filled with a filler such as **Stomahesive Paste**.
- 4 **Ileodress Plus Pouches** need not be removed while bathing, showering or swimming.
- 5 After exposure to water the textured back panel of the pouch should be patted dry with a towel.
- 6 Small amounts of adhesive residue which may remain upon changing the pouch, need not be completely removed

Preparing the Ileodress Plus Drainable Pouch for application


A Pre-cut holes

- 1 Measure the maximum diameter of the stoma and select a hole size which is the same size or a little larger than the base of the stoma.
- 2 Before positioning the pouch over the stoma check that there will be no gaps of exposed skin between the edge of the pre-cut hole and the base of the stoma.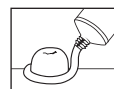

- 3 If there are likely to be exposed areas of skin, a filler such as Stomahesive Paste can be used around the stoma, to protect the skin before applying the pouch.

B The Starter hole

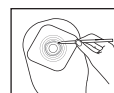- 1 Using the guidelines on the release paper on the back of the Stomahesive Disc, trace the outline of the stoma.

- 2 Before cutting the Stomahesive Disc, separate the front and back panels so as not to cut the front panel of the pouch.

- 3 Using a small pair of scissors, preferably curved, enlarge the starter hole to match the traced outline. Do not cut beyond the last circle on the guide.

Removing the pouch

Starting at the top, gently pull the pouch away from the skin. Apply light pressure on the skin, with your free hand, as you peel the pouch downwards.


Applying the Ileodress Plus Drainable Pouch

- 1 Remove the release paper from the skin barrier.

- 2 Hold the appliance by the sides and position the skin barrier with the hole over the stoma.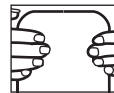


- 3 Press the Stomahesive Disc against the skin, paying particular attention to the area closest to the stoma. During application, maintain gentle pressure on the disc to improve adhesion.


- 4 Ensure that the skin barrier is smooth and secure around the stoma.


Fitting the clip to the Drainable Pouch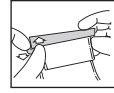


- 1 The clip must firstly be **fully** opened.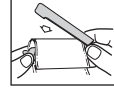

- 2 Hold the lower bar of the clip beneath the end of the pouch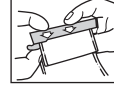

- 3 Fold the pouch over the bar **once** only. The pouch ends should lie comfortably exactly between the hinge and the opposite end of the clip.

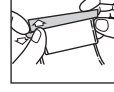- 4 Press down the closing bar of the clip and ensure it is securely closed.
- 5 To open the clip, push up the closing bar as shown.

Disposal of used pouches

- 1 Pouches should be emptied before disposal. They must not be flushed down the toilet as doing so may cause blockage of the sewage system.
- 2 Used pouches may be disposed of through the refuse collection service. They should be emptied, rinsed out and wrapped in newspaper or in a plastic bag and sealed securely before being placed in the dustbin.
3. Some local authorities have a soiled dressings collection service. Details of this service can be obtained from the local council offices or your General Practitioner.
4. Used equipment must not be burned on an electric fire or in an electric incinerator.

This device is for single-use only and should not be re-used.

Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

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© 2022 ConvaTec Inc.
ConvaTec Limited
UNOMEDICAL A/S
DKSH Taiwan Ltd.
ConvaTec (Australia) Pty Ltd.