




3 Press down the closing bar of the clip.

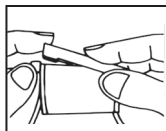

4 Make sure the clip is **securely closed**.

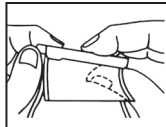


5 To open the clip, push away the closing bar as shown.

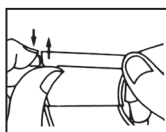

### Removing the Pouch

1 Starting at the top, gently pull the pouch away from the skin. Apply light pressure on the skin, with your free hand, as you peel the pouch downwards.

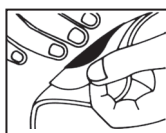

### Disposal of used pouches

- 1 Pouches should be emptied before disposal. They must **not be flushed down the toilet** as doing so may cause blockage of the sewage system.
- 2 Used pouches may be disposed of through the refuse collection service. They should be emptied, rinsed out, and wrapped in several layers of newspaper or in a plastic bag and sealed securely before being placed in the dustbin.
- 3 Some local authorities have a soiled dressings collection service. Details of this service can be obtained from the local council offices or your General Practitioner.
- 4 Used equipment must not be burned on a domestic fire or in an electric incinerator. Items which have been emptied and cleaned may be incinerated out of doors.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

# ILEODRESS

## One-Piece Drainable Pouch with Stomahesive Skin Barrier

Ileodress Drainable Pouches are one-piece appliances consisting of a Stomahesive Disc surrounded by a fabric support adhesive.

Specifically designed for people who need the protection of the Stomahesive Skin Barrier but who prefer a one-piece appliance, they offer the following advantages:

- 1 The unrivalled protection of the Stomahesive Skin Barrier.
- 2 A flexible fabric support adhesive that eliminates the need for 'picture framing'.
- 3 The specially textured barrier on the back panel of the pouch prevents moisture from accumulating thus increasing the comfort of the pouch where it touches the skin.
- 4 A soft, quiet odour barrier film.
- 5 Pre-cut hole sizes for ease of appliance, or a starter hole (19 mm) for those people who prefer to tailor the appliance to fit the stoma.
- 6 The choice of clear or opaque pouches.



Do not re-use



Medical Device



Caution



Importer



Consult instructions for use

For further information on the complete Product Range, please contact:

Manufactured by:  
ConvaTec Inc.  
Greensboro, NC 27409  
1-800-422-8811



UNOMEDICAL A/S  
Aaholmvej 1-3, Osted  
4320 LEJRE DENMARK



ConvaTec Limited  
First Avenue, Deeside Industrial Park  
Deeside, Flintshire, CH5 2NU, UK

United Kingdom 0800 282 254 - Ostomy Care  
www.convatec.com

© 2021 ConvaTec Inc.  
®/™ indicates a trademark of ConvaTec Inc.

Revised 2021-06  
1001022V1

en ENGLISH

# ILEODRESS®

## Drainable Pouches with Stomahesive® Skin Barrier



# ConvaTec

# ILEODRESS®

## One-Piece Drainable Pouch with Stomahesive Skin Barrier

### Choosing the correct hole size:

**A** Pre-cut holes are available in the following sizes.

	Standard Opaque Pouches	Standard Clear Pouches	Small Opaque Pouches
25mm (1")	Product Code <b>S832</b>	-	<b>S852</b>
32mm (1 1/4")	Product Code <b>S833</b>	-	<b>S853</b>
38mm (1 1/2")	Product Code <b>S835</b>	<b>S845</b>	<b>S855</b>
45mm (1 3/4")	Product Code <b>S836</b>	<b>S846</b>	<b>S856</b>
50mm (2")	Product Code <b>S837</b>	<b>S847</b>	<b>S857</b>
64mm (2 1/2")	Product Code <b>S840</b>	<b>S850</b>	<b>S860</b>

The diameter of the stoma should be measured and a hole size selected which is the same size or a little larger than the stoma.

**B** The Starter Hole ( Product Codes S831 Standard Opaque Pouch, S841 Standard Clear Pouch and S851 Small Opaque Pouch )

The Starter Hole is 19mm in diameter and can be cut to the same size and shape as the stoma, up to a maximum diameter of 64mm (2 1/2 inches).

### General Instructions


**Intended Use:** Management of stomal output.

**Indications for Use:** For use with an ileostomy.

- 1 Clean the skin around the stoma with soap and warm water and pat dry.
- 2 When using **Stomahesive Products** barrier creams and solutions are best avoided as they prevent good adhesion.
- 3 If the surface of the skin around the stoma is uneven due to scarring etc., such depressions may be filled with a filler such as **Stomahesive Paste**.
- 4 **Ileodress Pouches** need not be removed while bathing, showering or swimming.
- 5 After exposure to water, the fabric support adhesive and the textured back panel of the pouch should be patted dry with a towel.
- 6 Small amounts of adhesive residue which may remain upon changing the pouch, need not be completely removed.


### Preparing the ILEODRESS Drainable Pouch for application **A**

- 1 Measure the maximum diameter of the stoma and select a hole size which is the same size or a little larger than the base of the stoma.
- 2 Peel the protective plastic shield from the Stomahesive Disc.
- 3 Before positioning the pouch over the stoma check that there will be no gaps of exposed skin between the edge of the pre-cut hole and the base of the stoma.
- 4 If there are likely to be exposed areas of skin, a filler such as Stomahesive Paste can be used around the base of the stoma to protect this skin, prior to the application of the pouch.

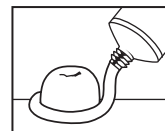

### Starter hole (Product Codes S831, S841, and S851) **B**

- 1 Peel the protective plastic shield from the Stomahesive Disc.
- 2 Using the guidelines on the release paper on the back of the Stomahesive Disc, trace the outline of the stoma.
- 3 Before cutting the Stomahesive Disc, separate the front and back panels so as not to cut the front panel of the pouch.
- 4 Using a small pair of scissors, preferably curved, enlarge the starter hole to match the traced outline. **Do not cut beyond** the last circle on the guide.

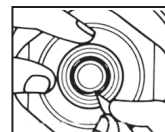

### Applying the ILEODRESS Drainable Pouch **A and B**




All product Codes

- 1 Remove the release paper from the skin barrier (a).
- 2 Remove the release paper from the fabric support adhesive (b) by bending the slip edges (c) sharply backwards.




**Do not** remove the release paper from the side edges (c) until later.

- 3 Hold the appliance by the two side edges (c) and position the skin barrier with the hole over the stoma.
- 4 Press the Stomahesive Disc against the skin, paying particular attention to the area closest to the stoma. During application, maintain gentle pressure on the disc to improve adhesion.
- 5 Press the fabric support adhesive to the skin, working outwards from the disc, making sure it is smooth and wrinkle free.
- 6 Remove the remaining release paper from the side edges (c) and smooth into place.

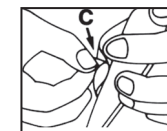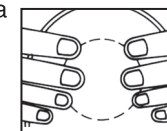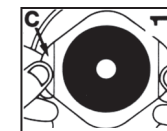


### Fitting the soft tie to the drainable pouch

- 1 Remove release paper from the adhesive strip on the tie. Place the tie about 1/4" (1 cm) from the end of the pouch, on the top side. **PRESS IN PLACE.**

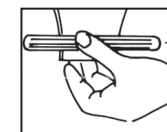


This can be done before applying the pouch to the body.

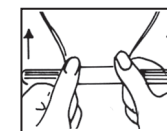

- 2 Tightly fold the pouch tail upwards 4 TIMES as shown.

- 3 Tightly fold the tie ends inwards and onto the underside of the pouch. Ensure that the pouch is smooth, flat and not crimped.

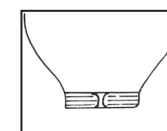

### Fitting the clip to the drainable pouch

- 1 The clip must firstly be **fully** opened.
- 2 Place the thin bar of the open clip on the top of the drainage outlet, fold the outlet over **once only** holding this end securely. Ensure that the drainage outlet is folded flat across the bar and lies **exactly** between the hinge and the lip at the opposite end of the clip.

