Natura™

High Output Drainable Pouch

Instructions for Use

MD

Medical Device

Do not re-use

i

Consult instructions for use

Caution

CH

REP

Abbreviation for authorised representative in Switzerland

FR

+

Sorting Information for Packaging – applicable to French territories only. Article 17 of the French AGEC Law and its implementing decree no. 2021-835 of 29 June 2021

Intended Use
Management of stomal output.

Indications for Use
For use with a colostomy or ileostomy.

For the management of a large volume of stomal effluent for individuals who require a high capacity pouch.


This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.


If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

FLANGE APPLICATION AND REMOVAL
Apply and remove the flange and pouch as indicated in the skin barrier flange instructions for use. This product may remain in place while bathing, showering, or swimming. Pat dry with a towel after exposure to water. Make sure the tap is closed prior to application.

FILTER REPLACEMENT
To eliminate and deodorize gas, the pouch incorporates a filter case with a water repellent filter inside. To replace the filter, remove the filter case cover, replace the old filter with a new filter (tan side facing out), and reseal the filter case.

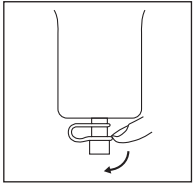USING THE OUTLET TAP
Remove the cap from the outlet tap and empty the pouch contents into the toilet or other approved container.

USING THE TAIL CLOSURE
For drainage of more solid stomal output:

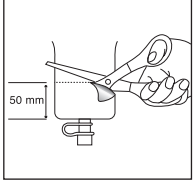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© 2025 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.
®/™ indique des marques déposées de ConvaTec Inc.
®/™ indica un marchio commerciale di proprietà di ConvaTec Inc.
®/™ geeft een handelsmerk van ConvaTec Inc. aan.