# Glide

with  
**FeelClean™**  
Technology

## en English

### Hydrophilic Intermittent Urinary Catheter

These instructions for use must be read and understood in parallel with the pictorial instructions for use also enclosed prior to using this device.

This device must only be used in a home healthcare setting, when prescribed by an appropriately qualified, trained healthcare professional. The patient or carer must follow the advice of the healthcare professional.

Hydrophilic Intermittent Urinary Catheters are intended to provide an intermittent pathway for drainage of fluids from the urinary bladder. The catheter is inserted through the urethra.

**Indications:** Intermittent drainage of the urinary bladder of those who need assistance with drainage due to conditions causing urinary retention or dysfunction of the urinary system.

**Users:** Males and Females

**Accessories:** Urine collection bags may be attached to the funnel end of the catheter before use.

**Duration of use:** The Catheters have been designed for routine intermittent use. Devices are single use only and must be removed immediately upon complete drainage of the bladder and disposed of. The overall usage of the device can be expected to be ongoing repeated multiple times a day.

**Contraindications:** Do not use in conjunction with irrigation or instillation.

**Warnings:**

- Check the catheter is the correct size as indicated on the packaging.
- Use the no-touch sleeve to minimize contact with the catheter.
- Use the catheter immediately upon extraction from the package.
- Only use sterile water supplied within the package for lubrication of the catheter.
- Always slide the catheter slowly and smoothly when inserting and removing. Never force the catheter.
- When removing the catheter, stop if more urine starts to flow. Continue again once urine stops flowing.
- The device is for single use only. Do not reuse or re-insert the device. This may lead to an increased risk of infection, transmission of viruses/bacteria, bleeding, tissue damage or pain.

**Contact a healthcare professional:**  
If there is no urine flow or other usage complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge).

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your competent authority.

**Storage and Disposal:**  
Store the product within the sealed packaging at a recommended room temperature of 10–25 °C (50–77 °F).

Sterility is maintained unless the sealed packaging is damaged or opened prior to use. Do not use the device if the packaging or the water sachet is damaged or open prior to use and dispose of the device according to local regulations.

**Technical assistance:**

If further information or guidance is needed visit us online at [www.gentlecathtech.com](http://www.gentlecathtech.com).

Additional paper copies of the Instructions for Use are available from ConvaTec Limited.

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ConvaTec Limited  
First Avenue, Deeside Industrial Park  
Deeside, Flintshire, CH5 2NU, UK  
[www.convatec.com](http://www.convatec.com)

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**EC REP:** UNOMEDICAL A/S  
Aaholmvej 1-3, Østed  
4320 LEJE DENMARK

TM indicates a trade mark of  
Unomedical a/s.  
©2022 ConvaTec Inc.  
Revised: 2022-01  
1719466V

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