AQUACEL® Pro Foam

PRODUCT DESCRIPTION
AQUACEL® Foam Pro dressing is a sterile, Hydrofiber™ foam wound dressing, consisting of:

- a waterproof, breathable outer polyurethane film which provides a bacterial and viral barrier
- a soft, multi-layered, absorbent central pad containing polyurethane foam and Hydrofiber™ (Sodium carboxymethylcellulose). The pad absorbs and retains moisture, exudates and bacteria
- a sterile silicone adhesive wound/skin contact layer to adhere the dressing to the patient

CLINICAL BENEFITS

AQUACEL® Foam Pro is designed to protect fragile skin from moisture, shear and friction damage with a gentle but effective adhesive silicone contact layer. AQUACEL® Foam Pro is also designed to provide a moist wound healing environment, whilst effectively managing excess exudate levels which may further damage the wound bed and surrounding skin.

INTENDED USE

AQUACEL® Foam Pro dressings may be used, with the consultation of a healthcare professional, for the management of exuding, non-exuding wounds and protection of intact skin.

AQUACEL® Foam Pro dressing may be included in a comprehensive protocol of care to protect intact skin against breakdown.

INDICATIONS

AQUACEL® Foam Pro dressing is indicated for the treatment of:

- Leg ulcers
- Pressure Ulcers
- Diabetic Foot Ulcers
- Surgical Wounds
- Traumatic Wounds

INTENDED USERS

AQUACEL® Foam Pro dressings are to be used by health care professionals, patients and carers.

TARGET PATIENT POPULATION

AQUACEL® Foam Pro dressings are intended to be used on patients with one of the wound types listed in the indications or those who require skin protection from breakdown.

CONTRAINDICATIONS

AQUACEL® Foam Pro dressings should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or components stated above.

PRECAUTIONS AND OBSERVATIONS

Sterility is guaranteed unless the pouch is damaged or opened prior to use. Do not use the device if the sterile barrier is damaged or opened prior to use and dispose of the device according to local regulations.

Do not use after stated expiry date.

AQUACEL® Foam Pro dressings are for single-use only, do not reuse. Reuse may lead to increased risk of infection and cross-contamination and delayed healing. If a dressing is used in addition to any other dressing, it is required to secure it in place. This dressing should not be used with other wound care products without first consulting a healthcare professional.

AQUACEL® Foam Pro dressings are water resistant however, should not be submerged during bathing or swimming.

AQUACEL® Foam Pro dressings are not compatible with oil-based products.

AQUACEL® Foam Pro dressings are not compatible with oxidising agents such as hydrogen peroxide or hypochlorite solutions.

A healthcare professional should be consulted if any of the following are observed during dressing changes, irritation (reddening, inflammation), maceration (whitening of skin), bleeding/garmling (excess tissue formation), signs of infection (increased pain, hyper-warmth/redness of surrounding tissue, wound exudate), or a change in wound colour and/or odour.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local state and federal laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

CONTACT AND DURATION OF USE

AQUACEL® Foam Pro dressings can be worn for up to 7 days, dressings should be changed earlier if clinically indicated, and are safe for continuous use.

DIRECTIONS FOR USE

- Storage**
 - Store at room temperature (10° C - 25° C/50° F - 77° F). Protect from light. Keep dry.

- Wound Site Preparation and Skin Cleansing:**
 - Before applying the dressing, cleanse the area with an appropriate wound cleanser and dry the surrounding skin.

3. Dressing Preparation and Application:

- When applying to intact skin choose a dressing which covers the entire vulnerable area.
- When applying to a wound choose a dressing size to ensure that the central absorbent pad is at least 1cm (1/3 inch) larger than the wound.
- Remove the dressing from the sterile pack.
- Remove the release liner, minimising finger contact with the adhesive surface.
- Hold the dressing over the wound or area of skin to be protected and line up the centre of the dressing with the centre of the wound or area of skin to be protected.
- Ensure the dressing is not stretched upon application. Smooth down the adhesive border. Ensure joints are sufficiently flexed to allow patient mobility.
- The dressing may be cut; additional tape or film dressing should be used to create a seal and ensure a bacterial barrier.

- Dressing Removal:**
 - The dressing should be changed when clinically indicated (the dressing should not be allowed to get over saturated) or if it becomes damaged/fouled.
 - To remove the dressing, press down gently on the skin and carefully lift one corner of the dressing, continue until all edges are free. Carefully lift away the dressing and discard as per local clinical protocols.
 - Only during preventative care, dressing corner can be lifted for inspection of skin and reassessed. Dressing must be changed if skin is broken.

Visit us online at www.convatec.com.

Made in UK

© 2026 ConvaTec Inc.

AQUACEL is a trademark of ConvaTec Inc. AQUACEL, the AQUACEL logo and Hydrofiber are registered trademarks

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