

# STOMAHESIVE

EN ENGLISH

## INSTRUCTIONS FOR USE



**STOMAHESIVE** is prepared from non-irritating ingredients, (gelatin, pectin, carboxymethylcellulose and polyisobutylene).

**STOMAHESIVE** is a protective wafer for use in the management of ileostomies, colostomies, ileal conduits and fistulae management.

**STOMAHESIVE** protects excoriated skin while healing takes place and provides a polythene surface to which appliances may be attached.

**STOMAHESIVE** will protect skin sensitive to plaster adhesives.

**STOMAHESIVE** will mould to the skin surface which may be irregular due to skin folds, scar tissue, etc.

**CAUTION:** Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

## Instructions for applying Stomahesive

**Intended Use:** Provide protection for peristomal skin by filling skin irregularities and depressions to improve adhesion of ostomy systems.


**Indications for Use:** For use with a colostomy, ileostomy or urostomy.



For maximum protection it is important that Stomahesive fits closely to the stoma. Using small scissors and a cutting guide, cut a centre hole to fit the stoma accurately.



Remove the backing paper from the Stomahesive.

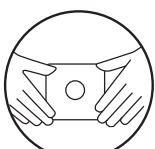

Place the Stomahesive over the stoma and smooth down.

## Hints for routine skin care when using Stomahesive

- Clean skin thoroughly and pat dry.
- Stomahesive should be fashioned to fit the stoma as closely as possible so that the skin around the stoma is completely protected.
- If the area around the stoma is irregular, which may permit leakage, Orabase or Stomahesive paste should be used as a filling before applying the Stomahesive square.
- Small amounts of adhesive residue which may be left behind after removal need not be completely removed prior to re-application.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

## Availability

	Pack Size	Ordering Code
Stomahesive* Wafers 100mm x 100mm	5	S100
Stomahesive* Wafers 200mm x 200mm	3	S101
Orabase* Paste	30g	S103
Orabase* Paste	60g	S104
Stomahesive* Paste	60g	S105
Orahesive* Powder	25g	S106
Varihesive* Wafers	5	S108

All these products are available on prescription form FP10



We pride ourselves on the quality of our products and the stringent testing they undergo before leaving our factory.

However, should you experience any difficulty with any of our products, please let us know.

© 2021 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.



Do not re-use



Medical Device



Consult instructions for use



Importer



Caution

ConvaTec Inc.  
Greensboro, NC 27409  
USA  
1-800-422-8811

ConvaTec Limited  
First Avenue, Deeside Industrial Park  
Deeside, Flintshire, CH5 2NU, UK

Sponsored in Australia by:  
ConvaTec (Australia) Pty Ltd.  
1800 339 412 Australia  
0800 441 763 New Zealand

UNOMEDICAL A/S  
Aaholmvej 1-3, Østved  
4320 LEJRE DENMARK

輸入販売元  
コニバテック ジャパン株式会社  
お客様相談窓口  
☎ 0120-532384

ConvaTec Polska Sp. z o.o.  
Al. Armii Ludowej 26,  
00-609 Warszawa,  
Polska

Linus Medical  
Duntes street 6-412,  
Rīga, LV-1013  
Latvia

ConvaTec Česká republika s.r.o.  
Olivova 4/2096  
110 00 Praha 1

Linus Medical  
Narva mnt 5,  
10117 Tallinn, Estonia

Versium, s.r.o.  
Seberního 2  
821 03 Bratislava  
Slovak Republic

Replant Cardio Kft.  
Nándorfejérvári u. 35,  
H-1119 Budapest  
Hungary

Импортер / Представитель в РФ:  
ЗАО «КонваТек»,  
115054, Москва,  
Космодамианская наб., д.52, стр.1.  
Тел.: +7 (495) 663 70 30  
Факс: +7 (495) 748 78 94  
Информационная линия поддержки  
КонваТек (звонок по России  
бесплатный): 800-200-80-99.  
Часы работы 9.00-17.00

STOMA MEDICAL d.o.o.  
Frana Folnegovića 1c  
10000 ZAGREB  
Phone: 00385 (0)1 5508 998  
Fax: 00385 (0)1 6177 217  
E-mail: rbeovic@stoma-medical.hr  
Web: www.stoma-medical.hr

PCP EOOD  
ул. „Виктор Григорович“ №3 ет.1,  
София 1606  
En. pošta: vsr@techno-link.com  
tel. (02) 851 90 70, 851 90 71  
факс: (02) 951 59 32  
Безплатна тел. линия: 0800 18 808

Zastopnik in distributer v Sloveniji:  
VALENCIA Stoma-Medical d.o.o.  
Gregorčičeva 9  
1000 Ljubljana  
Slovenija  
Brezplačna tel. številka: 080 15 45

Rombiomedica S.R.L.  
Str. Nicolai Gogol nr.1A,  
apart. 2, sector 1,  
011815 Bucharest  
Romania

ConvaTec Turkey  
S. İliknur Keleş Sk. Hüseyin  
Bağdatlıoğlu Plz. No: 5/3  
Kozyatağı, Kadıköy, 34742  
İSTANBUL TURKEY

Linus Medical UAB  
Ukmergės g. 223,  
LT-07156 Vilnius  
Lithuania

Australia	1 800 339 412
België	02 3899742 or 0800-12011
Bosna	080020208
Canada	1-800-465-6302
Česká republika	800122111
Chile	800-210-113
Črna Gora	0800 101 102
Danmark	48 167474
Deutschland	0800-78 66 200
Eesti	8003030
Egypt	(2012) 3645495 / (2012) 5557478
España	93 6023700
France	0800 35 84 80
Hong Kong / 香港	852-25169182
Hrvatska	01/5508 999 or 0800 8000
Ireland	1 800 721721
Israel	+ 972 3 9256712
Italia	800 930 930
Kosovo	044/ 116 722
Latvija	8000 5 333
Lietuva	8-800-70001
Luxemburg	+32 2 3899742 or 0800-23157
Magyarország	0680 201 201 (code: 1212)
Malaysia	1-800-880-601
Nederlând	0800-0224444 - stomazorgproducten 0800-0224460 - woundzorgproducten
New Zealand	0800-441-763
Norge	80030995
Österreich	0800-216339
Polska	0800 120 093
Portugal	800 20 1678
România	+40 21 230 23 90
Singapore	65-62459838
Slovenija	080 15 45
Slovensko	0 800 122 111
South Africa	+27 (0)861 888 842
Srbija	0800 101 102
Suisse	0800-551110
Suomi	0800 179 797
Sverige	08-704 72 50 / 020-21 22 22
Thailand	66-2-2240060
Türkiye	0216 416 52 00
United Kingdom	0800 282 254 - Ostomy Care 0800 289 738 - Wound Therapeutics

中國  
台灣 / 台湾  
한국  
日本  
Россия  
Україна  
Македонија  
България

www.convatec.com