Natura™ TM +

INSTRUCTIONS FOR USE - INSTRUCCIONES DE USO - MODE D'EMPLOI - INSTRUÇÕES DE USO - 使用説明 - 使用説明書 - GEBRAUCHSANWEISUNG - ISTRUZIONI D'USO - INSTRUCTIONS D'UTILISATION - GEBRUIKSIINSTRUCTIE - BRUKSANVISNING - KÄYTTÖOHJEET - BRUGSANVISNING - BRUKSANVISNING - ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ - INSTRUKCJA STOSOWANIA - HASZNÁLATI UTASÍTÁS - NÁVOD K POUŽITÍ - NÁVOD NA POUŽITIE - UPUTE ZA UPORABU - NAVODILA ZA UPORABO - ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ - INSTRUCIUNI DE UTILIZARE - KULLANIM TALİMATLARI - KASUTUSJUHEND - UPUTSTVO ZA UPOTREBU - LIETOŠANAS PAMĀCĪBA - NAUDOJIMO INSTRUKCIJS

ENGLISH

Natura™ ⊕ **Two-Piece Urostomy Pouch**

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Natura™ ⊕ Urostomy Pouch is part of a two-piece ostomy system. This pouch must be used with the same size Natura™, Natura™ ⊕, Sur-Fit Natura™, and Combihesive Natura™ skin barriers.

PRECAUTIONS AND OBSERVATIONS

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

INDICATION

For the management of stomal output from a urostomy.



DIRECTIONS FOR USE

Note: Refer to the skin barrier instructions for use for applying a pouch to the skin barrier, removing the pouch from the skin barrier and disposing the pouch.

1. Drainage of the Pouch

2. Connection to Night Drainage System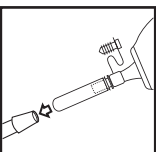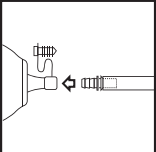

Note: Use only the connector provided.

Made in Dominican Republic

© 2023 ConvaTec Inc.

™ indicates a trademark of ConvaTec Inc. NATURA, Sur-Fit, and Sur-Fit Natura are registered trademarks in the United States.

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™ 顯示 trademarks of ConvaTec Inc. NATURA 是 ConvaTec Inc. 在美國的註冊商標
™ nurodo ConvaTec Inc. prekės ženklus. NATURA yra registruotas ConvaTec Inc. prekės ženklas Jungtinėse Valstijose.