ActiveLife®

One-Piece Urostomy Pouch with Stomahesive® Skin Barrier

Instructions for Use

Single Use

Read instructions for use.

For the management of stomal effluent.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

APPLICATION


The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

FOR PRECUT POUCH PREPARATION



Select a pouch with the **precut stoma opening** the same size as the largest diameter of the stoma. Do not enlarge the precut stoma opening.

FOR CUSTOM CUT POUCH PREPARATION

Using the guide on the release paper, trace the outline of the stoma.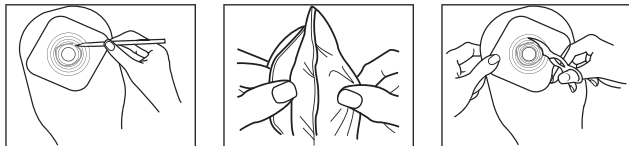


APPLICATION

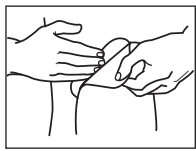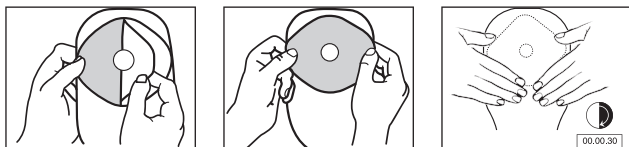
Ensure a snug fit around the stoma.

REMOVAL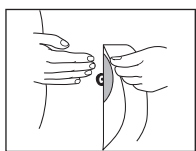


USING THE FOLD OVER DRAINAGE TAP

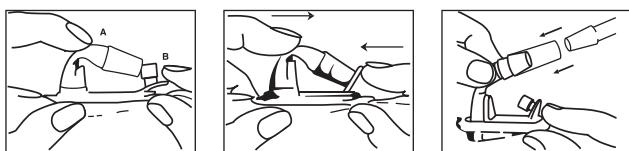This product may remain in place while bathing, showering, or swimming.

Adhere to local disposal requirements/regulations.

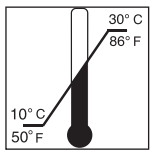Storage temperature.

Keep dry. Avoid high humidity.

Lot/ control number

Expiry date

Should irritation occur, consult a healthcare professional.

For more information on ConvaTec products, contact us at the address below.

Manufactured by:

ConvaTec Inc.
Greensboro, NC 27409
1-800-422-8811

ConvaTec Limited, Deeside, CH5 2NU, UK

www.convatec.com

© 2024 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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Convatec Inc.
Convatec Limited, Deeside, CH5 2NU, UK
© 2024 Convatec Inc.
MD/MC est une marque déposée de Convatec Inc.