**One-Piece Drainable Pouch****Instructions for Use**

CE

 Medical Device
  Contains biological material of animal origin
  Importer

 Do not re-use
  Consult instructions for use
**INTENDED USE**

Management of stomal output.

**INDICATIONS FOR USE**

For use with a colostomy or ileostomy.

**CAUTION:** Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

**POUCH PREPARATION**

The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

**For pouches with a precut stoma opening:**

Select a pouch with the precut stoma opening at least 1/8 in. (3mm) **larger** than the largest stoma diameter.

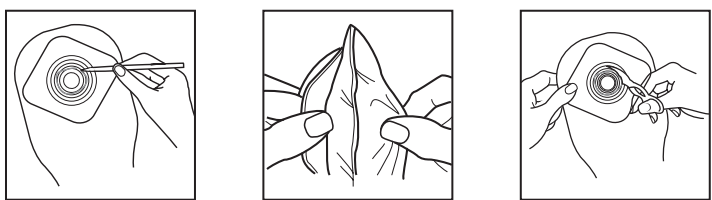
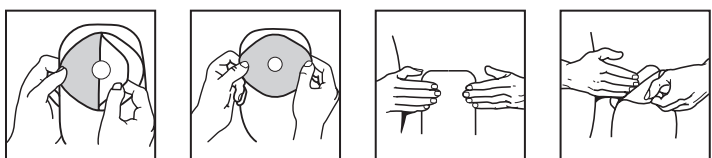
**DO NOT enlarge the precut stoma opening.**

Remove protective plastic shield from Stomahesive® disc.

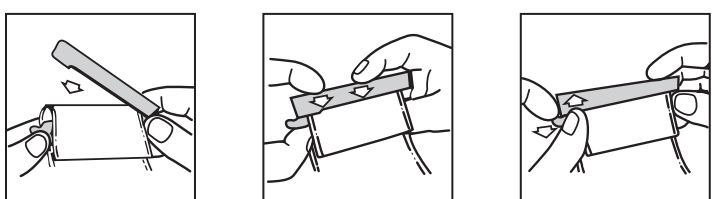
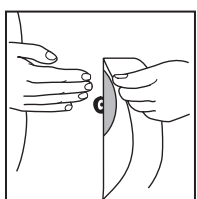
**For custom cut pouches:**

Before cutting, separate front and back panels.

Using guidelines on release paper, cut the center hole the same size as the stoma. **Do not cut beyond last line.**

**POUCH APPLICATION**

Remove release papers. Ensure a snug fit around the stoma.

**USING THE TAIL CLOSURE****POUCH REMOVAL**

This product may remain in place while bathing, showering, or swimming. Pat dry after exposure to water.

**DISPOSAL**

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

**STORAGE CONDITIONS**
 Temperature limit

 Keep dry

 Caution

**Note:** Should irritation occur, discontinue use and consult a healthcare professional.

© 2022 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec (Australia) Pty Ltd.