Combihesive® Natura®

Stomahesive® Flange
Stomahesive® Flexible Flange
Stomahesive® Flange
with Flexible Collar

Instructions for Use

 ConvaTec

	Do not re-use
	Caution
	Consult instructions for use
	Medical Device
	Importer
	Contains biological material of animal origin

STORAGE CONDITIONS

 Temperature limit

 Keep dry

 Batch code

Intended use: Management of stomal output.

Indication for Use: For use with a colostomy, ileostomy or urostomy.


PRECAUTIONS AND OBSERVATIONS


CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.


Note: Should irritation occur, consult a healthcare professional.

FLANGE PREPARATION


The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

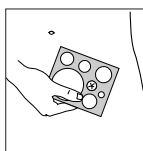 Select a flange size that is 6 - 10 mm **larger** than the widest diameter of the stoma.

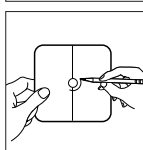 Using the cutting guide, trace the exact stoma outline onto flange release paper.

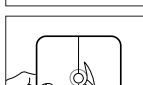
 Cut along tracing leaving at least 6 mm of skin barrier between the enlarged starter hole and the flange ring. Remove release paper.

FLANGE APPLICATION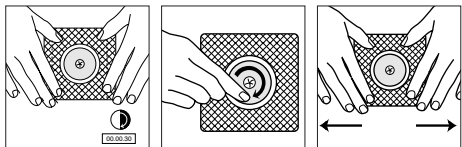


Ensure a snug fit around the stoma.

POUCH APPLICATION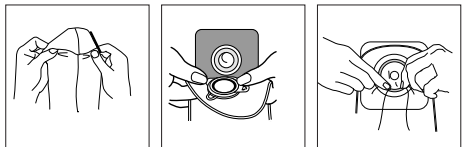


This product may remain in place while bathing, showering, or swimming. For closed pouches with a filter, cover filter with filter covers provided. Pat dry after exposure to water.

POUCH REMOVAL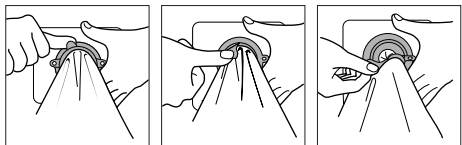

FLANGE REMOVAL

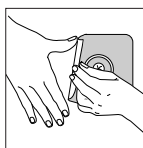USING THE INVICLOSE™ OUTLET DIRECTIONS TO CLOSE THE POUCH:

The InvisiClose tail closure system (**1**) is comprised of a security flap (**A**), inter-locking closures (**B**) and outlet end-strips (**C**).

 Hold the security flap against the front of the pouch revealing the inter-locking closures. (**2**)

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
Fold the tail end up towards you four times and ensure the inter-locking closures overlap. **(3)**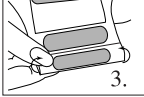

Press the inter-locking closures together until you feel them lock (CLICK) into place along their entire length. **(4)**


Wrap the security flap around to the back of the pouch (body side) and press the inter-locking closures together until you feel them lock (CLICK) into place. **(5)**

FOR DRAINING THE POUCH:

Turn the back of the pouch (body side) towards you. Carefully lift the security flap up towards you to unlock. **(6)**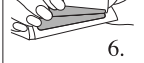

While holding the security flap up against the front of the pouch, unlock the inter-locking closures by lifting at the corner **(7)** and unraveling the folds four times. **(8)**


To facilitate draining and/or cleaning, you may open the tail end by pushing in both ends of the outlet end-strips with your thumb and forefinger. **(9)**


If desired, pouch tail may be cleaned with gentle soap and water. Pat dry.


USING THE TAIL CLOSURE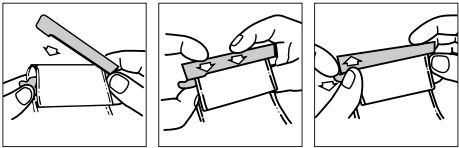


USING THE ACCUSEAL® TAP

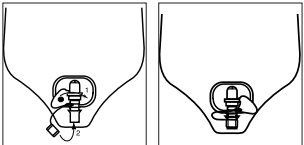
For connection to a night drainage system: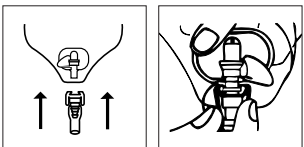

USING THE FOLD OVER DRAINAGE TAP

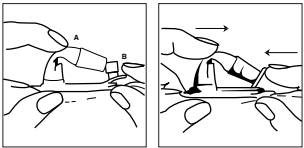For connection to a night drainage system:

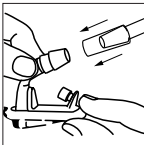DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

For more information on ConvaTec products, contact us at the address below.

© 2022 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Inc. Greensboro, NC 27409 USA 1-800-422-8811	ConvaTec Limited First Avenue, Deeside Industrial Park Deeside, Flintshire, CH5 2NU, UK
Sponsored in Australia by: ConvaTec (Australia) Pty Ltd. 1800 339 412 Australia 0800 441 763 New Zealand	UNOMEDICAL A/S Aaholmvej 1-3, Østed 4320 LEJRE DENMARK
Imported by: DKSH Taiwan Ltd. 10th Floor, No. 22, Lane 407, Tiding Blvd., Sec. 2, Neihu Technology Park, Taipei City 114, Taiwan, R.O.C.	ConvaTec International Services GmbH Mühlentalstrasse 36/38 8200 Schaffhausen, Switzerland
Australia Belgie Bosna Canada Česká republika Chile Danmark Deutschland Eesti Egypt España France Hong Kong / 香港 Hrvatska Ireland Israel Italia Kosovo Latvija	輸入販売元 コバテック ショパン株式会社 お客様相談窓口 ☎ 0120-532384
1800 339 412 02 3899742 or 0800-12011 080020208 1-800-465-6302 800122111 800-210-113 48 167474 0800-78 66 200 8003030 (2012) 3645495 / (2012) 5557478 93 6023700 0800 35 84 80 852-25169182 01/6121 624 / 0800 8000 1 800 721721 + 972 3 9256712 800 930 930 044/ 116 722 8000 5 333	

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