ConvaTec

Combihesive Natura®

Disposable Convex Insert / Konvexe
Einlageringe / Bagues-Confort convexes /
Inserti Convessi / Aandrukringen

Instructions for Use / Gebrauchsanweisung /
Conseils d'utilisation / Istruzioni per l'uso /
Gebruiksaanwijzing

en ENGLISH   

To improve contact between the skin barrier and the peristomal skin area around a flush or recessed stoma, a stoma with an adjacent depression, scar, or crease, or other conditions for which additional barrier-to-skin contact is desirable. Use of the convex insert may result in increased wearing time of the flange.

CAUTION

Use caution when there is a known circulatory compromise to the stoma or area adjacent to peristomal skin.

IMPORTANT: This product with deep convexity should only be used after prior assessment of suitability by an appropriate medical professional.

Intended Use: To add convexity to two piece ostomy systems.


Indications for Use: For use with a colostomy, ileostomy or urostomy; and with compatible ConvaTec flat two piece ostomy systems.

APPLICATION

PRIOR to flange application:

1. If necessary, enlarge flange stoma opening (see product instructions).

2. Select a convex insert with an inner diameter at least 7 mm **larger** than the maximum diameter of the stoma.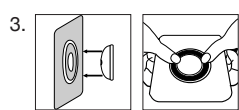

Press the convex insert securely into the ring.

NOTE: If necessary, Durahesive® flanges can be prestretched by pushing down on the flange area **inside** the ring. The release paper will tear allowing for easy insertion.

4. Remove release paper and apply flange as directed in product instructions. Exposed skin between stoma and flange may be filled with Stomahesive® Paste (see product instructions).


DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

STORAGE CONDITIONS

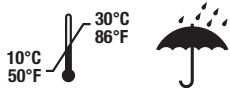MD Medical Device

Note: Should irritation occur, consult a healthcare professional.

For more information on ConvaTec products, contact us at the address below.

© 2021 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Limited
DKSH Taiwan Ltd.
ConvaTec (Australia) Pty Ltd.