# ActiveLife®

## One-Piece Drainable Pouch

### Instructions for Use

## III ConvaTec

For the management of stomal output.

**CAUTION:** Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

#### POUCH PREPARATION

The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

#### For pouches with a precut stoma opening:

Select a pouch with the precut stoma opening at least 1/8 in. (3mm) **larger** than the largest stoma diameter.

#### DO NOT enlarge the precut stoma opening.

Remove protective plastic shield from Stomahesive® disc.


#### For custom cut pouches:

Measure stoma with guide provided. Remove protective plastic shield from Stomahesive® disc.

Before cutting, separate front and back panels.


Using guidelines on release paper, cut the center hole the same size as the stoma. **Do not cut beyond last line.**

#### POUCH APPLICATION

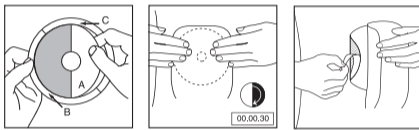Remove release papers B and C. Ensure a snug fit around the stoma.

#### USING THE TAIL CLOSURE
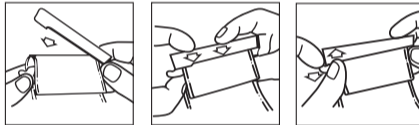
#### POUCH REMOVAL


This product may remain in place while bathing, showering, or swimming. Pat dry after exposure to water.

#### DISPOSAL

Adhere to local disposal requirements/regulations.

#### STORAGE CONDITIONS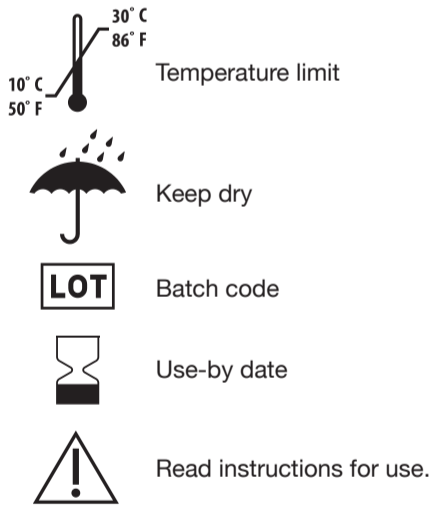

Note: Should irritation occur, discontinue use and consult a healthcare professional. For more information on ConvaTec products, contact us at the address below.

© 2022 ConvaTec Inc.  
®/™ indicates a trademark of ConvaTec Inc.

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®/™ são marcas da ConvaTec Inc.
ConvaTec Limited, First Avenue
ConvaTec (Australia) Pty Ltd.
DKSH Taiwan Ltd.