eakin Cohesive®

seal | anillo | anneau


eakin Cohesive®
large / grande / grand
REF 839001

eakin Cohesive®
small / pequeño / petit
REF 839002

eakin Cohesive® slims™
REF 839005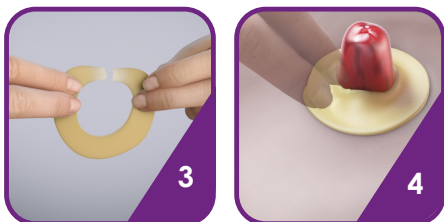

eakin Cohesive® seals

en

eakin Cohesive® seals are moisture absorbing skin protecting seals which protect peristomal or periwound skin from harmful attack by aggressive body fluids by creating a protective gel. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

Indications for use:

- An ostomy seal
- A sealing barrier for intact skin around fistula and wounds.
- An external packing agent on uneven skin

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Instructions for use

Application

1. Remove the seal from its outer package.
2. Ensure that the area for application is clean, dry and free from traces of grease or cream.
3. Warm the seal in your hands for 20 – 30 seconds until soft before removing the protective discs.

Gently stretch the eakin Cohesive® seal as required to fit the application site. It may also be cut and securely re-joined to fit around more difficult sites.

4. Apply to the skin with gentle pressure and hold in place until the eakin Cohesive® seal is secure on the skin. Mould the seal towards the base of the stoma or the edge of the wound.
5. Ensure a snug fit, with no gaps around the base of the stoma, to provide maximum skin protection. Ensure the fit is not too tight around the stoma, which might prevent normal ostomy function.
6. Place any appliance directly onto the surface of the eakin Cohesive® seal. Hold in place with gentle pressure until securely attached to the seal and the surrounding skin.

Removal

Gently remove any appliance from the body. Remove the eakin Cohesive® seal by carefully peeling and rolling from the outer edge. Wipe gently to remove any remaining protective gel from the skin. A medical adhesive remover can be used if necessary.

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eakin Cohesive® stomawrap™

en

eakin **Cohesive®** stomawrap™ are moisture absorbing skin protecting seals which protect peristomal or periwound skin from harmful attack by aggressive body fluids by creating a protective gel. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

Indications for use:

- An ostomy seal
- A sealing barrier for intact skin around fistula and wounds
- An external packing agent on uneven skin

Intended users

For ostomates, individuals managing fistulae or wounds, healthcare professionals and caregivers.

Instructions for use

Application

1. Remove the seal from its outer package.
2. Ensure that the area for application is clean, dry and free from traces of grease or cream.
3. Warm the seal in your hands for 20 – 30 seconds

until soft before removing the protective discs. Gently stretch the eakin **Cohesive®** stomawrap™ as required to fit the application site. eakin **Cohesive®** stomawrap™ is pre-cut, making it easier to fit around large, oval or irregular-shape stomas.

4. Apply to the skin with gentle pressure and hold in place until the eakin **Cohesive®** stomawrap™ is secure on the skin. Mould the seal towards the base of the stoma or the edge of the wound.
5. Overlap the two ends of the product and ensure a snug fit, with no gaps around the base of the stoma, to provide maximum skin protection. Ensure the fit is not too tight around the stoma, which might prevent normal ostomy function.
6. Place an appliance directly onto the surface of the eakin **Cohesive®** stomawrap™. Hold in place with gentle pressure until securely attached to the seal and the surrounding skin.

Removal

Gently remove any appliance from the body. Remove the eakin **Cohesive®** stomawrap™ by carefully peeling and rolling from the outer edge. Wipe gently to remove any remaining protective gel from the skin.

A medical adhesive remover can be used if necessary.

Disposal

Dispose of waste products in accordance with local guidelines.

Warnings and cautions

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any of these symptoms, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Do not re-use. Re-use of a single use product may cause harm to the user due to the risk of contamination and impairment in product function.

Other information

Store flat in the carton at room temperature between 10° and 30° Celsius (50° and 86° Fahrenheit). Do not refrigerate.

eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®'s current recommendations.

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S122108 | Feb 2022 | ©2022 TG Eakin Ltd