
Intermittent Urinary Catheter Male / Intermittierender Harnkatheter für Männer / Catetere urinario intermittente maschile / Cathéter urinaire intermittent pour homme / Catéter urinario intermitente masculino / Cateter urinário intermitente masculino / Ανδρικός Καθετήρας Διαλειπόντων Καθετηριασμών

Instructions for use / Gebrauchsanweisung / Istruzioni per l'uso / Notice d'utilisation / Instrucciones de uso / Instruções de uso / Οδηγιών χρήσης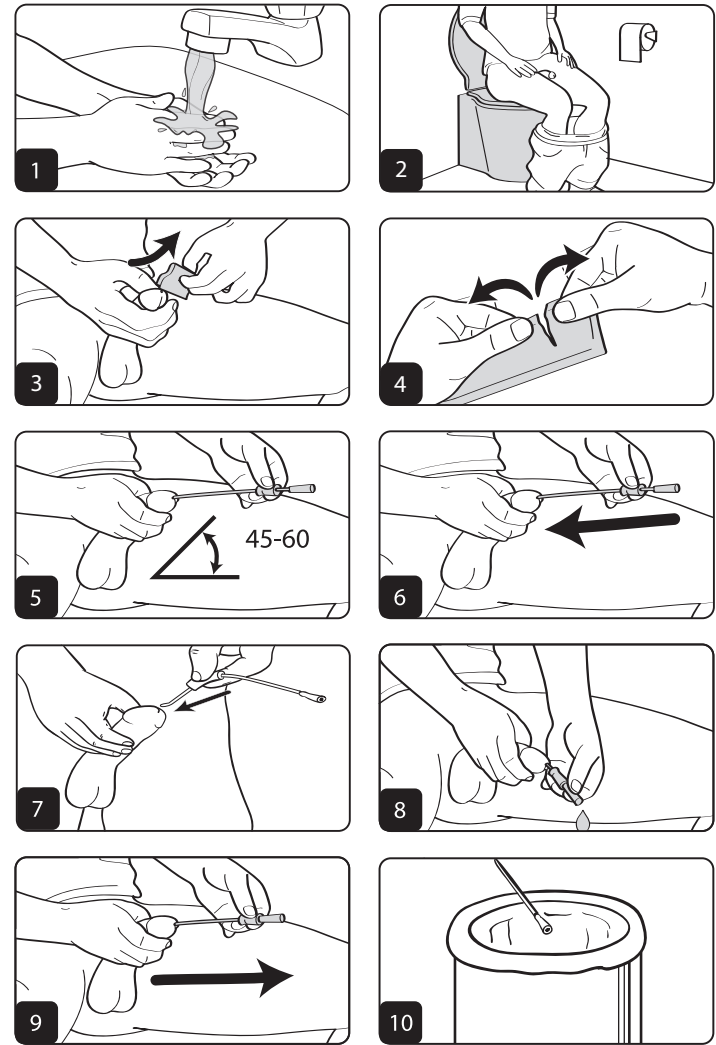

ENGLISH

These instructions for use must be read and understood in parallel with the pictorial instructions for use also enclosed prior to using this device. The individual steps in the pictorial instructions, showing the preparation and performance of catheterization, must be followed.

The device must only be used when prescribed by an appropriately qualified, trained healthcare professional. The patient or carer must follow the advice of the healthcare professional.

Cure Ultra® Pre-lubricated Catheters are intended to provide an intermittent pathway for drainage of fluids from the urinary bladder. The catheter is inserted through the urethra.

Indications for use: The Cure Ultra® is a pre-lubricated intermittent urinary catheter that is inserted through the urethra and indicated for the purpose of bladder drainage (for males and females). The urinary catheter comes in a variety of sizes and is packaged sterile for single-use.

Users: Males

Accessories: Urine collection bags may be attached to the funnel end of the catheter before use.

Duration of use: The Catheters have been designed for routine intermittent use. Devices are single-use only and must be removed immediately upon complete drainage of the bladder and disposed of. The overall usage of the device can be expected to be ongoing repeated multiple times a day.

Environment: professional healthcare facilities, homes, work, public bathrooms, out and about.

Contraindications: Do not use in conjunction with irrigation or instillation.

Warnings:

- The catheter must be used immediately once removed from the pouch.
- The catheter is lubricated and ready-to-use once removed from the packaging. The catheter should never be re-lubricated.
- Check if the package is unopened or undamaged. Do not use if package is damaged.
- Check the catheter is the correct size as indicated on the packaging.
- Use the no-touch sleeve or the gripper sleeve to minimize contact with the catheter, if available.
- Always slide the catheter slowly and smoothly when inserting and removing. Never force the catheter.
- When removing the catheter, stop if more urine starts to flow. Continue again once urine stops flowing.
- The device is for single use only. Do not reuse or re-insert the device. This may lead to an increased risk of infection, transmission of viruses/bacteria, bleeding, tissue damage or pain.
- Coudé /Tiemann tip: The catheter tube features a blue guide stripe down the length of the catheter, which helps the user visualize which way the insertion tip is pointing during catheterization.

Caution: Federal law restricts this device to sale or on the order of a physician. * US only

Contact a healthcare professional:

- If there is no urine flow or other usage complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge).

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your competent authority.

Storage and Disposal:

Store the product within the sealed packaging at ambient temperature.

Sterility is guaranteed unless the sealed packaging is damaged or opened prior to use. Do not use the device if the packaging is damaged or open prior to use and dispose of the device according to local regulations.

Technical Assistance:

If further information or guidance is needed, visit us online at <https://curemedical.com>. Additional paper copies of the Instructions for Use are available from Convatec Inc.

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Puede solicitar copias impresas adicionales de las instrucciones de uso a Convatec Inc.