DuoDERM[®] EXTRA THIN

STERILE R

EXTRA THIN CGF[®] SPOTS

For the protection and management of superficial dermal wounds

III ConvaTec

PRODUCT DESCRIPTION

DuoDERM Extra Thin Spots are highly flexible, control gel formula dressings designed for use on dry to lightly exuding wounds. DuoDERM Extra Thin Spots are particularly suitable over sutured post-operative wounds, excision sites, and minor abrasions.

DuoDERM Extra Thin Spots interact with wound moisture producing a soft mass that enables removal of the dressing with little or no damage to newly formed tissues. They help isolate the wound against bacterial and other external contamination.

INDICATIONS

- Post-operative wounds – excision and biopsy sites.
- Minor abrasions.
- Management of superficial, dry to lightly exuding dermal ulcers.
- Protective dressings.

CONTRAINDICATION

Use on individuals with a known sensitivity to the dressing.

PRECAUTIONS AND OBSERVATIONS

When Used on Dermal Ulcers:

1. Initial use of this product should be under the direction of a health professional.

Infection: If signs of clinical infection should develop, such as: uncharacteristic odor or change in the color of the exudate, fever or cellulitis (tenderness and erythema in the area of the wound), a bacterial culture of the wound site should be taken. If clinical signs of infection are present, appropriate medical treatment should be initiated. DuoDERM dressings may be continued during the treatment at the discretion of the clinician.

PREPARING AND CLEANSING THE WOUND SITE

DuoDERM Extra Thin Spots are sterile and should be handled appropriately.

Assure that the DuoDERM Extra Thin Spot will extend beyond the wound margin. Cleanse the wound according to hospital practice. Irrigate with saline and dry the surrounding skin to ensure it is free of any greasy substance.

APPLYING THE DRESSING

- 1) Remove backing paper, minimizing finger contact with adhesive surface.

- 2) Apply spot to wound. Smooth into place particularly around the edges.

REMOVING THE DRESSING

- 1) Press down on the skin and carefully lift an edge of the dressing. Continue around until all edges are free.

Repeat cleansing procedure. It is unnecessary to remove all residual dressing material from the surrounding skin.

Leave the dressing in place (not more than 7 days) unless it is uncomfortable, leaking, or there are clinical signs of infection.

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HOW SUPPLIED

DuoDERM® Extra Thin CGF® Dressing,
Sterile Individually Wrapped

	Order No.
1.75 x 1.5 Spots, Box of 20	187932
4 x 4, Box of 10	187955
6 x 6, Box of 10	187957
2 x 4, Box of 20	187900
2 x 8, Box of 10	187961
3 x 3, Box of 20	187901
4 x 6 oval, Box of 10	187902
6 x 7 triangle, Box of 10	187903

OTHER PRODUCTS

DuoDERM® CGF® Control Gel Formula Dressing, Sterile, Individually blister packed	
4 x 4, Box of 5	187660
4 x 4, Box of 20 (Institutional Pack)	187658
6 x 6, Box of 5	187661
6 x 6, Box of 20 (Institutional Pack)	187659
6 x 8, Box of 5	187643
8 x 8, Box of 5	187662
8 x 12, Box of 5	187644
DuoDERM® CGF® Control Gel Formula Border Dressing, Sterile	
2.5 x 2.5, Box of 5	187970
4 x 4, Box of 5	187971
6 x 6, Box of 5	187972
4 x 5 triangle, Box of 5	187973
6 x 7 triangle, Box of 5	187974
DuoDERM® Hydroactive® Dressing, Sterile	
4 x 4, Box of 5	187610
4 x 4, Box of 20 (Institutional Pack)	187611
8 x 8, Box of 3	187620
DuoDERM® Hydroactive® Paste Sterile, 30 gram tube	187930

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Temperature limit / Límite de temperatura /
Limites de température

ConvaTec Inc.
Greensboro, NC 27409
USA
1-800-422-8811

 ConvaTec Limited, First Avenue,
Deeside Industrial Park, Deeside,
Flintshire, CH5 2NU, UK

Imported by:
DKSH Taiwan Ltd.
10th Floor, No. 22, Lane 407,
Tiding Blvd., Sec. 2,
Neihu Technology Park, Taipei City 114,
Taiwan, R.O.C

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd.
1800 339 412 Australia
0800 441 763 New Zealand

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© 2024 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.
de ConvaTec Inc.