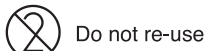
# ILEODRESS®

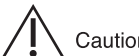
## Drainable Pouch

## One-Piece Ostomy System

### Instructions for Use

Do not re-use

Caution

Contains biological material of animal origin

**Intended Use:** Management of stomal output.

**Indications for Use:** For use with an ileostomy.

**CAUTION:** Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

#### POUCH PREPARATION

The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

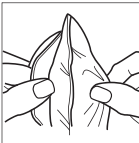
#### FOR PRECUT POUCH PREPARATION

Select a pouch with the precut stoma opening the same size as the largest diameter of the stoma diameter. **DO NOT enlarge the precut stoma opening.** Remove protective plastic shield from Stomahesive® disc.


#### FOR CUSTOM CUT POUCH PREPARATION

Measure stoma with guide provided. Remove protective plastic shield from Stomahesive® disc. Using the guide on the release paper, trace the outline of the stoma.


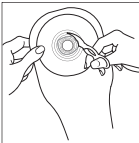
Before cutting, separate front and back panels.




Cut along tracing. **Do not cut beyond last line.**

#### POUCH APPLICATION

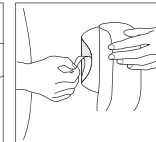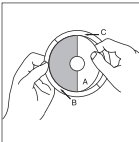Remove release papers B and C. Ensure a snug fit around the stoma.

#### USING THE TAIL CLOSURE
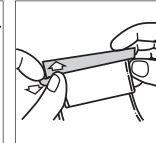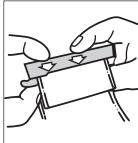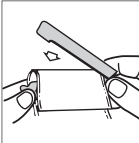
#### POUCH REMOVAL

This product may remain in place while bathing, showering, or swimming. Pat dry after exposure to water.

#### DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

#### STORAGE CONDITIONS

Medical Device

Importer

Note: Should irritation occur, consult a healthcare professional.

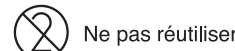
For more information on ConvaTec products, contact us at the address below.

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© 2022 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.