
- Applying the Skin Barrier (Figure 2)
- Remove release liner from the skin barrier.
- Center the opening of the skin barrier over the stoma.
- Apply the skin barrier against the skin around the stoma for 30 seconds.
- Remove or remaining release liner from the adhesive.
- Smooth in place and remove wrinkles.
- Wear as instructed and avoid secure.Wear time may vary.

4. Applying the Pouch
  - Separate the pouch flaps to allow air to enter. (Figure 3)
  - Hold the Accordion Flange™ and the pouch flange. (Figure 4)
  - Line up the Accordion Flange™ and the pouch flange. (Figure 4)
  - Starting at the bottom, apply gentle pressure around the circumference of the flange until they "snap" in place and feel secure.
  - Try to tug the pouch to ensure a secure connection. (Figure 5)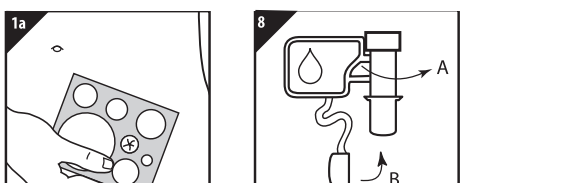

5. Emptying the Pouch
  - Drainable Pouches with InvisiClose™ "Tall Closure" (Figure 1)
  - Field the tall closure beginning with the outlet end strip until the interlocking closures overlap. Press the interlocking closures together across the full length and zipper until you feel them close. If desired, turn the pocket inside-out to conceal the tall closure. (Pocket is available on Natura™ pouches only.)
  - Drainable Pouches with Tall Closure (Figure 7)
  - Drainable Pouches with Accordion Flange™ (Figure 8)
  - Drainable Pouches with Fold over Top (Figure 9)
  - Drainable Pouches with Soft Top (Figure 10)

6. Emptying the Pouch
  - Drainable Pouches with InvisiClose™ "Tall Closure" (Figure 11)
  - Unlock the pouch closure and flip up by pushing in both ends of the outlet end strip with fingers. (Figure 12)
  - Drainable Pouches with Tall Closure (Figure 12)
  - Drainable Pouches with Accordion Flange™ (Figure 13)
  - Drainable Pouches with Fold over Top (Figure 14)
  - Drainable Pouches with Soft Top (Figure 14)

7. Connection to Night Drainage System
  - Accusafe™ with Soft Top (Figure 16)
  - Soft Top (Figure 17)

8. Information about Pouches with Filters
  - Cover the filter with the sticker provided when bathing, swimming, etc.
  - Remove after water contact.

9. Removing the Pouch (Figure 19)

10. Removal of the Skin Barrier (Figure 19)

11. Disposal
 

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimized for intended use.

12. Disposal
 

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and/or your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

13. Disposal
 

© 2022 Convatec Inc.  
 \*Indicates trademarks of Convatec Inc.

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20. Disposal
 

© 2022 Convatec Inc.  
 \*Andrzejewski Watercare/Convatec Inc.

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