ESENTA™

Skin Barrier

(Foam Applicator)

 **ESENTA™ Skin Barrier (Foam applicator)**

Sting free skin barrier

INTENDED USE: Protection of skin from the damage associated with body waste, enzymes and adhesives.

DESCRIPTION OF DEVICE & CONTENTS: Sting free skin barrier

Indications: For use on skin around stomas and skin around wounds.

TARGET PATIENT POPULATION: People with stoma and wound dressings.

INTENDED USERS: Ostomates, Patients, Health Care Professionals and Caregivers.

Instructions for use:

- Clean and thoroughly dry the skin.
- Tear sachet open and remove foam applicator.
- Gently wipe over the area to be protected.
- Ensure ESENTA™ skin barrier is completely dry before applying adhesive products, such as appliance or wound dressings.
- Re-application is recommended every 12 to 72 hours, or at each dressing change.

Caution:

- Use only as directed.
- Do not use on open wounds
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt patch test a small area before use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

- Sterility is guaranteed unless pack is damaged or opened prior to use. Do not use the device if packaging is damaged or open prior to use and dispose of the device according to local regulations.
- If during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

- After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

 **Warning!** Highly flammable – keep away from sources of ignition. No smoking. Keep out of reach of children. Store at room temperature (10°-25°C / 50°-77°F).

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