STOMAHESIVE®

Powder / Пудра / Puder / Pudr / Pulber / Tapadópor / Püderis / Pudra / Пупра / Puder / Pudră / Puder / Zásyp / Posip

ENGLISH

Intended use: Intended to absorb moisture and protect skin from damage, and to facilitate the application of ostomy systems.

Indications for use: For use with a colostomy, ileostomy or urostomy.

Caution:

Do not use with patients with known sensitivity to Stomahesive® Powder or its constituents.

Directions for use:

- Wash hands and dry well.
- Apply the powder as a thin layer as often as necessary. Gently puff the powder onto the moist area. If any of the powder lands on dry skin gently wipe away with a damp cloth. If the area is very moist you may have to repeat this process once or twice. Once completed it is safe to apply the powder directly. Do not apply the powder to the baseplate/wafer.
- Keep cap tightly closed.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

Waste Disposal

Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

Precautions and warnings:

Do not apply the adhesive powder directly under the ostomy system support as this may lead to a delay in adherence.

© 2023 ConvaTec Inc.

*™ indicates a trademark of ConvaTec Inc.

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