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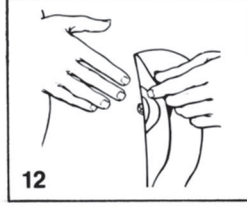
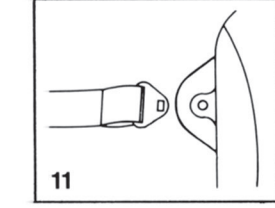
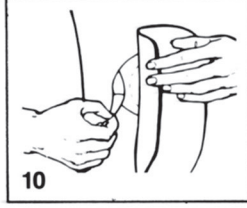
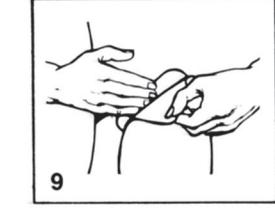
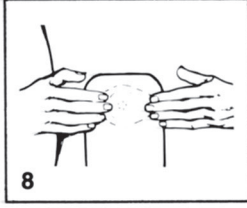
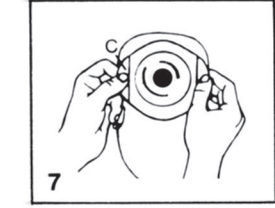
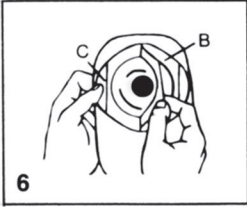
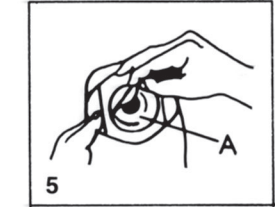
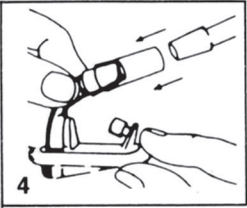
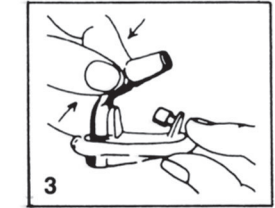
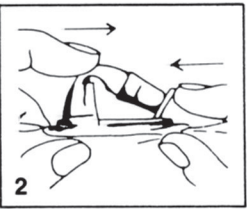
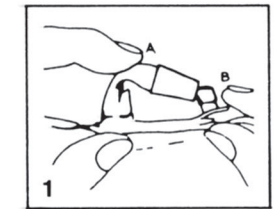
ENGLISH — Instructions for use

PRODUCT DESCRIPTION

Active® Life Convex One-piece Urostomy Pouch with Durahesive® Skin Barrier provides increased convexity around the stoma. Belt tabs are incorporated as part of the pouch. The picture frame support adhesive around the skin barrier eliminates the need for taping. The urostomy pouch is constructed with a nonreturn valve so that urine can flow freely into the collection area of the pouch but cannot flow backward toward the stoma.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management. If a leakage occurs or your skin around your stoma is red, broken and/or itching, please consult your ostomy nurse or health care professional for advice.

USING THE DRAINAGE TAP

NOTE: Close the drainage tap prior to pouch application.

1. Close drainage tap by bending outlet spout (A) over to meet plug (B).
2. Press plug fully into spout.
3. To open tap, hold pouch end up, squeeze outlet tube immediately behind plug and pull plug from outfit.
4. To attach to a night drainage container, use the adapter provided. The Sur-Fit® Night Drainage Container is available.

PREPARATION AND APPLICATION

Cleanse the peristomal area with water and pat **thoroughly** dry. Your hands and the skin surrounding the stoma must be clean, dry and free from oily substances before applying to ensure adhesion is not impaired.

IMPORTANT - Select an Active Life Convex Pouch with the pre-cut stoma opening the same size as the largest diameter of the stoma. **Do not enlarge the precut opening.**

5. Remove the plastic shield from the Durahesive® disc (A).
6. Remove the release paper from the picture frame support adhesrve (B) by bending the slit edges (C) sharply backwards. Do **NOT** remove the release paper from the side edges (C) until later.
7. Hold the pouch by the two side edges (C) keeping it in an upright manner and away from the body until the center of the Durahesive disc has been positioned over the stoma.
8. Press the Durahesive disc against the skin, paying particular attention to the area closest to the stoma.
9. Press the picture frame support against the skin, making sure it is smooth and wrinkle free.
10. Remove the remaining release paper from the side edges (C) and smooth into place.

NOTE: When Stomahesive® Paste is indicated, apply a ring of paste to the back (adhesive side) of the Durahesive disc, around the precut opening. Allow the paste to set for one minute prior to application to the skin.

BELT ATTACHMENT

11. If a belt is used, attach it to the belt tabs on the pouch and adjust the belt to fit comfortably.

POUCH REMOVAL

12. Gently pull a corner of the adhesive collar with one hand while gently applying pressure to the adjacent peristomal skin with the other hand. A water soluble adhesive remover is recommended if the pouch is difficult to remove.

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© 2024 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.
ConvaTec Inc.
®/™ indica marcas de ConvaTec Inc.
MD/MC est une marque déposée de ConvaTec Inc.