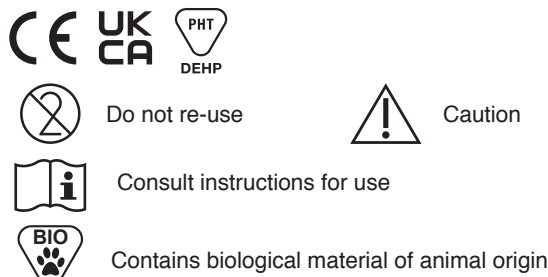
Visi-Flow®

Irrigation System

Instructions for Use

PRODUCT DESCRIPTION

For persons with colostomies for whom irrigation is indicated for evacuation of stool from the colon.

Intended Use: Irrigation of the stoma.

Indications for Use: For use with a colostomy.

WARNINGS AND PRECAUTIONS

- This irrigation method may not be suitable for all stoma types. Check with your health care professional before you begin the irrigation process to determine the best irrigation method for your stoma type.
- Ensure the stoma cone is clean and undamaged before insertion to help prevent injury to the stoma.
- Consult a healthcare professional if persistent bleeding from the stoma is experienced during or after irrigation.
- Persons receiving pelvic or abdominal radiation should consult with their healthcare professional prior to irrigation.

PREPARING AND FILLING THE IRRIGATOR BAG

1. Attach stoma cone to tubing. Note: If your healthcare professional has recommended the catheter method of irrigation, attach the appropriate catheter instead of the stoma cone

2. Roll the flow regulator control up until the tubing is completely closed prior to filling.

3. Open the top of the irrigator bag by separating and unsnapping the handles. Place the irrigator bag in the sink and fill it with lukewarm water in the amount prescribed by your healthcare professional.

Snap handles closed. Check the water level in the irrigator bag by holding it in an upright position with the top of the bag closed.

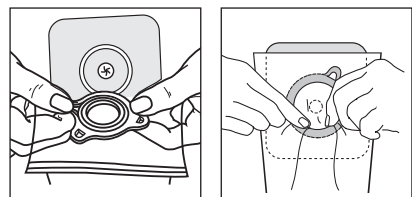
4. Hang the irrigator bag so that it is high enough to allow adequate water flow. If questions arise, contact your healthcare professional.

Height Reference Point


USING THE IRRIGATOR WITH THE IRRIGATION SLEEVE

Apply flange as directed in instructions for use provided with flange at **least one hour prior** to beginning the irrigation procedure.

Attaching the Sleeve


Fitting a Belt (if required)

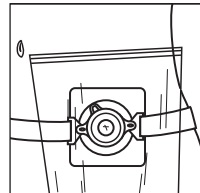INSTRUCTIONS FOR THE IRRIGATION PROCESS

IMPORTANT: Before starting the irrigation process:

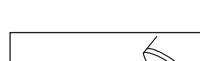- Sit on the toilet or on a chair next to the toilet.
- Position the bottom of the irrigation sleeve so it hangs into the toilet bowl.
- Remove entrapped air from the tubing by opening the flow regulator to allow a small amount of water to run through the stoma cone into the toilet.
- Close the flow regulator.

1. Apply a small amount of water soluble lubricant (medical grade) on the stoma cone.

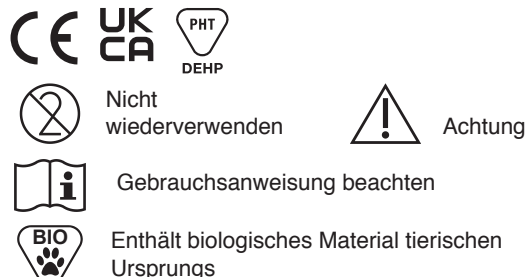
2. Open the top of the irrigation sleeve and position the stoma cone into the stoma using gentle pressure.

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
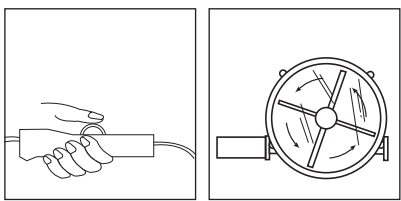
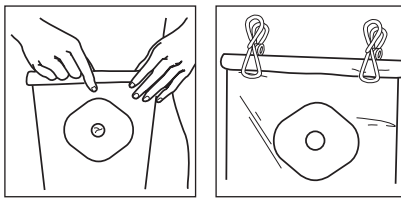
3. Holding the stoma cone in the stoma, open the flow regulator and allow water to flow into the colon. The rate of flow can be adjusted by using the flow regulator. The flow indicator allows a visual check of the rate of flow. If cramping is experienced, do not remove the stoma cone, but reduce or stop the flow of water and gently massage the abdomen until cramps subside. Then open the flow regulator so that the water will again enter the colon slowly.


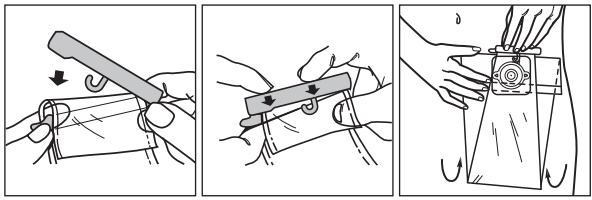
4. After the prescribed amount of water has entered the colon, stop the water flow by rolling the flow regulator closed. Remove the cone from the stoma. Fold the top of the sleeve over several times and seal using the spring clips. Use brush supplied with starter kit to clean stoma cone.


5. After **initial** evacuation of stool, attach the tail closure to the tapered bottom of the irrigation sleeve and hook to release tab of sleeve.

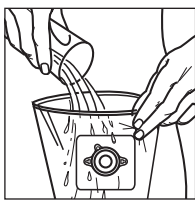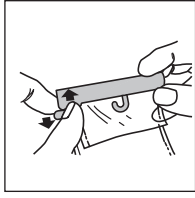To open tail closure:

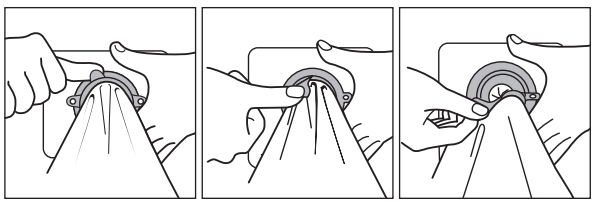6. After evacuation is complete, rinse the sleeve by running water through the top opening.

7. Remove the irrigation sleeve from the flange by pulling the tab located at the top of the sleeve ring. Maintain gentle pressure on the flange as the sleeve is removed.

8. Remove the flange as directed in instructions for use provided with flange. If you experience difficulty removing the flange, use the adhesive remover as directed in instructions for use provided with adhesive remover.

DISPOSAL

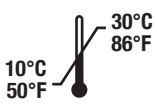
Do not flush components down the toilet.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

STORAGE CONDITIONS

MD Medical Device

Note: Should irritation occur, consult a healthcare professional.

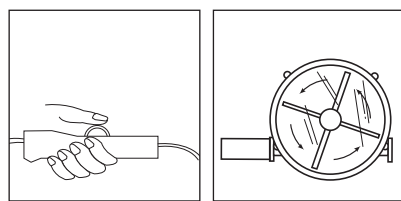
For more information on ConvaTec products, contact us at the address below.

EN

This product contains the phthalate DEHP. Phthalates are widely used as plasticizers in medical products and have undergone extensive testing. However, when using the device, it should be taken into consideration that possible health effects when used for long durations in pregnant women, nursing mothers and children cannot be entirely discounted.

PHT Device contains phthalate: bis(2-ethylhexyl)phthalate (DEHP)

© 2022 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Inc. Greensboro, NC 27409 1-800-422-8811 **EC REP** UNOMEDICAL A/S Aaholmvej 1-3, Osted 4320 LEJRE DENMARK
UK ConvaTec Limited First Avenue, Deeside Industrial Park Deeside, Flintshire, CH5 2NU, UK
CH REP ConvaTec International Services GmbH Mühlentalstrasse 36/38 8200 Schaffhausen, Switzerland
© 2022 ConvaTec Inc. Revised 2022-03
® Een handelsmerk van ConvaTec Inc. 1095416V2