Hydrophilic intermittent urinary catheter
 These instructions for use must be read and understood in parallel with the pictorial instructions prior to using this device. The individual steps in the pictorial instructions, showing the preparation and performance of catheterisation, must be followed.

The device must only be used when prescribed by an appropriately qualified, trained healthcare professional. The patient or carer must follow the advice of the healthcare professional.

Intended Use: Hydrophilic Intermittent Urinary Catheters are intended to provide an intermittent pathway for drainage of urine from the urinary bladder. The catheter is inserted through the urethra.

Indications: Intermittent drainage of the urinary bladder of users who need assistance with drainage due to conditions causing urinary retention or dysfunction of the urinary system.

Intended User: Male and Female Users, HCPs & Lay Caregivers.

Accessories: Urine collection bags may be attached to the funnel end of the catheter before use.

Duration of use: The catheters have been designed for routine intermittent use. Devices are single use only and must be removed immediately upon complete drainage of the bladder and disposed of. The overall usage of these devices can be expected to be ongoing, repeated multiple times a day.

Intended Environment: Professional Healthcare Facilities, Homes, Work, Public Bathroom

Contraindications

- Do not use in conjunction with irrigation or instillation.

Warnings

- Check the catheter is the correct size as indicated on the labelling.
- Avoid contact with the outer surface of the catheter tube.
- The catheter is lubricated and ready to use when the blue insertion tip cap is removed. The catheter should never be re-lubricated.
- Once the blue insertion tip cap is removed the catheter must be used immediately.
- Always slide the catheter slowly and smoothly when inserting and removing. Do not use excessive force when using the catheter.
- Once urine starts to flow, do not insert further than necessary.
- When removing the catheter, stop if more urine

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es, US

Information

- Coudé tip (US only): The catheter tube features a blue guide stripe down the length of the catheter and a fin on the funnel which can assist the user in alignment. Keep the blue stripe and the angled tip facing the front of you e.g. towards your nose whilst you slide the catheter slowly into the opening of the urethra and into the bladder.
- Nelaton / Straight tip: The catheter tube requires no orientation during insertion into the urethra and into the bladder.

Contact a healthcare professional:

- If there is no urine flow or other usage complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge).

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your competent authority.

Storage and Disposal

- Sterility is guaranteed unless the pouch is damaged or opened prior to use.
- Do not use the device if the packaging is damaged or open prior to use.
- Dispose of the device according to local regulations.

Technical assistance

If further information or guidance is needed, please visit us online at www.convatec.com

Additional paper copies of the Instructions for Use are available from Convatec Limited.

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© 2025 Convatec Inc.
The TM indicates a trademark of the Convatec Group