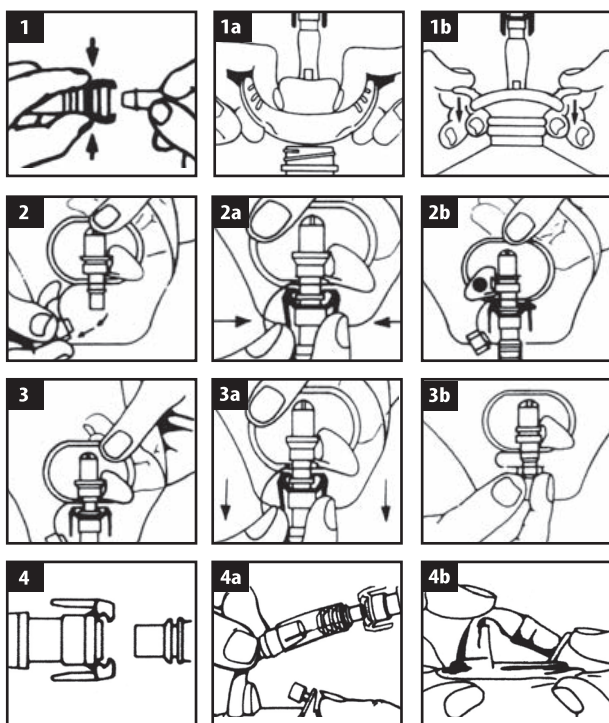
CE SUR-FIT[®] system

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SUR-FIT[®] NIGHT DRAINAGE CONTAINER SET

PRODUCT DESCRIPTION

The Sur-Fit Night Drainage Container Set is comprised of a plastic graduated container (2000 cc capacity), tubing with an attached Accuseal[®] connector and a cloth cover. The container set is designed for use with the Sur-Fit[®] Urostomy Pouch with Accuseal[®] Tap. However, also included is a universal adapter for attaching the tubing to a regulat SurFit[®] Urostomy Pouch or other urostomy pouches.

GENERAL INSTRUCTIONS

1. Keep the tubing from twisting or kinking so the urine can easily flow into the container.
2. Wash and rinse the drainage container and tubing daily with mild soap and water
3. Prior to connecting the tubing to a pouch for night drainage, a small amount of water may be added to the container to keep it in one position.
4. Remove the protective cap from the Accuseal connector at the free end of the tubing by pulling apart while squeezing the green arms, prior to attachment to a pouch. **(FIG. 1)**

ATTACHING THE CAP TO THE CONTAINER (FIG. 1A-1B)

1. Roll the side tabs and bottom lip of the cap back towards the tubing.
2. Rest the cap in the container opening.
3. Roll the side tabs down until the cap snaps into place
4. To remove the cap, pull back the side tabs.

Note: The tubing can be removed from cap by pulling apart while squeezing the small green arms.

ATTACHING TO A SUR-FIT UROSTOMY POUCH WITH ACCUSEAL TAP (FIG. 2-2B)

1. With the pouch's swivel valve in the closed position, remove the plug from the tap.
2. Position the Accuseal connector of the tubing over the tap stem and push until they snap in place.
3. Turn the swivel valve to the open position. The red dot should be facing away from the body.

REMOVING THE CONTAINER SET FROM A SUR-FIT UROSTOMY POUCH WITH ACCUSEAL TAP (FIG. 3-3B)

1. Close the swivel valve on the pouch, The red dot should be facing toward the body.
2. While holding the drainage assembly, pull apart, squeezing the small green arms.
3. Replace the plug into the tap stem.

ATTACHING THE CONTAINER SET TO A SUR-FIT UROSTOMY POUCH USING THE UNIVERSAL ADAPTER (4-4B)

Note: The universal adapter may also be used to connect the container set to other types of urostomy pouches.

1. Position the connector parts as shown and push until they snap in place.
2. Attach the open tubing to the drainage tap of the urostomy pouch.
3. To remove, gently pull the drainage tap of the pouch from the universal adapter.
4. Close the drainage tap.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

© 2024 ConvaTec Inc.

^{®/™} indicates a trademark of ConvaTec Inc.

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Sorting Information for Packaging – applicable to French territories only. Article 17 of the French AGEC Law and its implementing decree no. 2021-835 of 29 June 2021 /

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ConvaTec Inc.
Greensboro, NC 27409
USA
1-800-422-8811

ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

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Imported by:
DKSH Taiwan Ltd.
10th Floor, No. 22, Lane 407,
Tiding Blvd., Sec. 2,
Neihu Technology Park, Taipei City 114,
Taiwan, R.O.C.

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd
1800 339 412 Australia
088 441 763 New Zealand

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