Surgicare

Urihesive™ strips

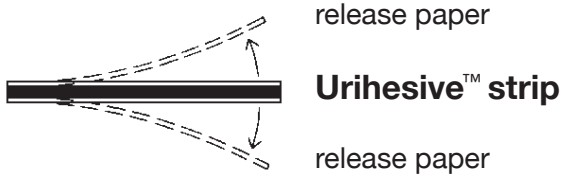
15 strips 120mm x 15mm

(en) to aid the management of urinary incontinence

Urihesive™ strips are prepared from non-sensitising ingredients including gelatin, pectin, sodium carboxymethylcellulose and polyisobutylene.

Urihesive™ strips make the management of urinary incontinence in men (using condom type devices) more acceptable by providing comfortable leak free adhesion of the condom to the penis avoiding the risk of irritation and soreness often associated with other types of adhesive.

Applying the Urihesive™ strip



release paper

Urihesive™ strip

release paper

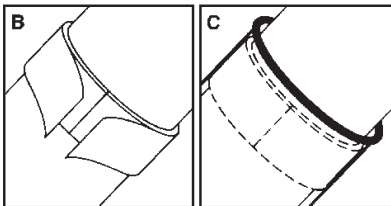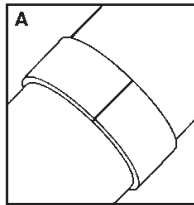1. Remove the hair at the base of the penis if necessary.
2. Cut the **Urihesive™ strip** to a length which will just meet when wrapped around the penis.
3. Expose the adhesive surface on **one** side of the **Urihesive™ strip** by removing the release paper and apply the **Urihesive™ strip** around the penis **(A)**. Firm adhesion is ensured if the strip is held in place for about 15 seconds.

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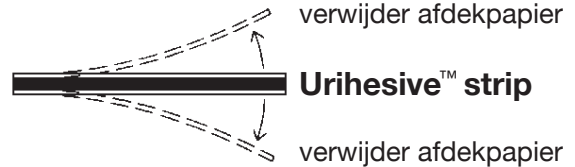
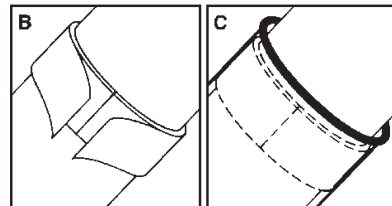
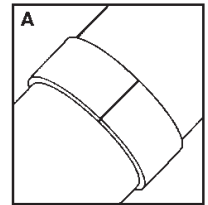
4. Remove the release paper from the outer surface of the **Urihese™ strip (B)**.
5. **Remove any traces of talc or other protection from the inside of the condom.**
6. Roll on the urine collecting device so that the collar comes behind the **Urihese™ strip** and squeeze onto the **Urihese™** surface to ensure a secure seal **(C)**. It is important that the collecting device is of suitable diameter and length so that creasing on top of the **Urihese™ strip** does not occur and that twisting of the device below the end of the penis is avoided.
7. The **Urihese™ strip** should be replaced each day when the condom is changed.
8. Any residue of the **Urihese™** remaining on the skin after removing the strip may be cleaned away with soap and water.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

 Medical Device  Importer  Authorized representative in the European Community

Order No. S120

For further information on the complete **ConvaTec Product Range** please contact

Manufactured by:

ConvaTec Inc.

Greensboro, NC 27409

 ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

 UNOMEDICAL A/S
Aaholmvej 1-3, Osted
4320 LEJRE DENMARK

United Kingdom
0800 282 254 - Ostomy Care
0800 289 738 - Wound Therapeutics

Ireland 1 800 721721
Israel + 972 3 9256712

© 2021 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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®/™ Een handelsmerk van ConvaTec Inc.