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Instructions for use / Instrucciones de uso / Mode d'emploi / 使用说明

en One-Piece Urostomy Custom-cut Pouch with Stomahesive® Skin Barrier


PRODUCT DESCRIPTION

The ConvaTec Little Ones™ Drainable pouch is a one-piece pouch with an adhesive skin barrier. Specifically designed for pediatric use, this pouch offers the following advantages:


1. Similar skin protective qualities as those offered by the Stomahesive® skin barrier, but modified to be gentle to skin.
2. A starter hole that can be enlarged to accommodate stomas of any size and shape up to 2" (50mm) in diameter.
3. A fabric-like pouch back panel for increased comfort.
4. A soft, quiet odor barrier film.
5. A soft, moldable tail closure designed for pediatric use.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.


PRODUCT PREPARATION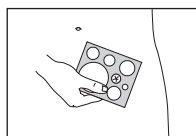


1. Measure stoma with guide provided. Gently cleanse the area with water and pat thoroughly dry. Peristomal skin **MUST** be free of any solvent or greasy substance.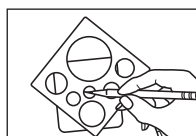

2. Trace stoma size onto the white backing paper, centering around the starter hole.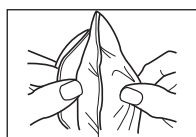

3. **IMPORTANT:** Before cutting, separate the front and back panels so as not to cut the front panel of the pouch.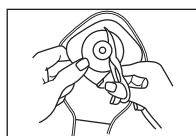


4. Using curved scissors, enlarge the center hole slightly larger than the tracing. **DO NOT** cut beyond the circle on the release paper.


ATTACHMENT OF THE TAIL CLOSURE

IMPORTANT: (a) Applying the tail closure **as described** is important to prevent leakage.


- (b) Affix the tail closure prior to application of the pouch to the body.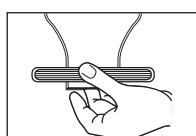

5. Remove the release paper from adhesive strip on tail closure. Position the closure approximately 1/4" from end of pouch tail, on the side of the pouch **without** the skin barrier. Press in place.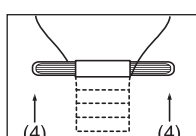


6. Tightly fold the pouch tail upward **4 TIMES** as shown.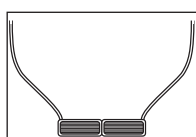


7. Tightly fold tail closure ends as shown toward the side of the pouch **with** the skin barrier, ensuring the tail is **smooth, flat, and not crimped**.

POUCH APPLICATION


IMPORTANT: Care should be taken to exert only gentle pressure on the abdomen of infants when applying any pouching system.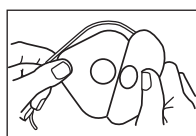

8. Remove the release paper from the skin barrier, minimizing finger contact with the adhesive. **NOTE:** When Stomahesive® Paste is used, it is recommended that it be applied as a ring around the enlarged hole on the adhesive side of the skin barrier.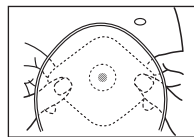


9. Hold the two sides of the skin barrier and position the enlarged hole over the stoma.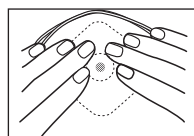

10. Gently press the barrier against the skin, paying particular attention to the area closest to the stoma.

The pouch may be worn for as long as it is comfortable and there are no signs of irritation.

POUCH REMOVAL

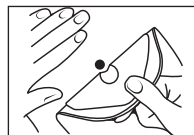11. Removal can be accomplished from top to bottom by gently pulling the skin barrier away from the skin while applying light pressure on the adjacent skin with your free hand. It is not necessary to remove all residue.

© 2023 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Limited, First Avenue