


CE COLODRESS PLUS

ONE PIECE STOMA CAP

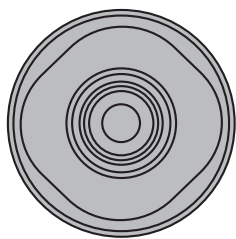


INSTRUCTIONS FOR USE

COLODRESS PLUS

ONE PIECE STOMA CAP Product Description

The Colodress Plus Stoma Cap is designed for use by patients with a permanent end colostomy. It provides a cover for the stoma but has no capacity, and is therefore useful for wear over a short period of time or for people who irrigate and do not require the security of a closed pouch.

The Stoma Cap has an all-over skin barrier of Stomahesive, and is equipped with a filter for odour elimination or reduction and gas ventilation.

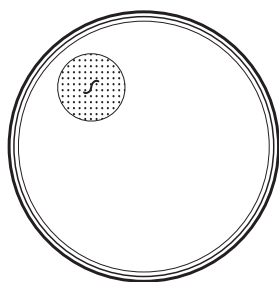

How to apply the Colodress Plus Stoma Cap

- The Stoma Cap may be cut to fit stomas up to a maximum of 50mm (2 inches).

The release paper has been printed with sizes 25mm, 32mm, 38mm, 45mm and 50mm to assist this process.
- Using a small pair of scissors, preferably curved, enlarge the starter hole to match the stoma size.
DO NOT CUT BEYOND THE 50mm GUIDE LINE
- Once the correct size has been cut out, remove the release paper and position the cap over the stoma pressing gently around the area to secure the adhesive.

Disposal of used Stoma Caps

- Caps must **not be flushed** down the toilet as doing so may cause blockage of the sewage system.
- Used caps may be disposed of through the refuse collection service. They should be wrapped in newspaper or in a plastic bag and sealed securely before being placed in the dustbin.
- Some local authorities have a soiled dressings collection service. Details of this service can be obtained from the local council offices or your General Practitioner.
- Used equipment must not be burned on a domestic fire or in an electric incinerator.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

COLODRESS PLUS

ONE PIECE STOMA CAP Ordering Information

	Product Code
Colodress Plus Stoma Cap	S821
Pack of 30	

All orders should be addressed to:
Sales Service Department, **ConvaTec Limited**, Unit 20, 1st Avenue,
Deeside Industrial Park, Deeside, CH5 2NU.

Do not re-use Consult instructions for use Caution Importer MD Medical Device

ConvaTec Inc.
Greensboro, NC 27409
USA
1-800-422-8811

ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd.
1800 339 412 Australia
0800 441 763 New Zealand

UNOMEDICAL A/S
Aaholmvej 1-3, Osted
4320 LEJRE DENMARK

輸入販売元
コンバテック ショパン株式会社
お客様相談窓口
☎ 0120-532384

ConvaTec Polska Sp. z o.o. Al. Armii Ludowej 26, 00-609 Warszawa, Polska	Linus Medical Duntes street 6-412, Riga, LV-1013 Latvia
ConvaTec Česká republika s.r.o. Olivova 4/2096 110 00 Praha 1	Linus Medical Narva mnt 5, 10117 Tallinn, Estonia
Versium, s.r.o. Seberiniho 2 821 03 Bratislava Slovak Republic	Replant Cardo Kft. Nándorfejérvári u. 35, H-1119 Budapest Hungary
Импортер / Представитель в РФ: ЗАО «КонваТек», 115054, Москва, Космодамианская наб., д.52, стр.1. Тел.: +7 (495) 663 70 30 Факс: +7 (495) 748 78 94 Информационная линия поддержки КонваТек (звонок по России бесплатный): 800-200-80-99. Часы работы 9.00-17.00	STOMA MEDICAL d.o.o. Frana Folnegovića 1c 10000 ZAGREB Phone: 00385 (0)1 5508 998 Fax: 00385 (0)1 6177 217 E-mail: rbgovic@stoma-medical.hr Web: www.stoma-medical.hr
PCP EOOD ул. „Виктор Григорович“ №3 ет.1, София 1606 Ел. поща: rsr@techno-link.com тел. (02) 851 90 70, 851 90 71 факс: (02) 951 59 32 Безплатна тел. линия: 0800 18 808	TT Medik Bulevar Mihajla Pupina 10 D/1 11070 Novi Beograd Srbija
Rombiomedica S.R.L. Str. Nicolai Gogol nr.1A, apart. 2, sector 1, 011815 Bucharest Romania	ConvaTec Turkey S. ilknur Keles Sk. Huseyin Bagdatlioglu Plz. No: 5/3 Kozyatagi, Kadikoy, 34742 ISTANBUL TURKEY
Linus Medical UAB Ukmergės g. 223, LT-07156 Vilnius Lithuania	

Australia	1 800 339 412
België	02 3899742 or 0800-12011
Bosna	080020208
Canada	1-800-465-6302
Česká republika	800122111
Chile	800-210-113
Crna Gora	0800 101 102
Danmark	48 167474
Deutschland	0800-78 66 200
Eesti	8003030
Egypt	(2012) 3645495 / (2012) 5557478
España	93 6023700
France	0800 35 84 80
Hong Kong / 香港	852-25169182
Hrvatska	01/5508 999 or 0800 8000
Ireland	1 800 721721
Israel	+ 972 3 9256712
Italia	800 930 930
Kosovo	044/ 116 722
Latvija	8000 5 333
Lietuva	8-800-70001
Luxemburg	+32 2 3899742 or 0800-23157
Magyarország	0680 201 201 (code: 1212)
Malaysia	1-800-880-601
Nederland	0800-0224444 - stomazorgproducten 0800-0224460 - woundzorgproducten
New Zealand	0800-441-763
Norge	80030995
Österreich	0800-216339
Polska	0800 120 093
Portugal	800 20 1678
România	+40 21 230 23 90
Singapore	65-62459838
Slovenia	080 15 45
Slovensko	0 800 122 111
South Africa	+27 (0)861 888 842
Srbija	0800 101 102
Suisse	0800-551110
Suomi	0800 179 797
Sverige	08-704 72 50 / 020-21 22 22
Thailand	66-2-2240060
Türkiye	0216 416 52 00
United Kingdom	0800 282 254 - Ostomy Care 0800 289 738 - Wound Therapeutics 4006065576
中国	0800-886-989
台灣 / 台湾	82-2-34536333
한국	☎ 0120-532384
日本	+7 800 200 80 99
Россия	+380 44 451 85 51
Україна	02/ 3 248 426 / 02/ 3 248 458
Македонија	0800 18 808
България	

www.convatec.com

© 2021 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.