STOMAHESIVE®

(en) ENGLISH

INSTRUCTIONS FOR USE

STOMAHESIVE® is prepared from non-irritating ingredients, (gelatin, pectin, carboxymethylcellulose and polyisobutylene).

STOMAHESIVE® provides a polythene surface to which appliances may be attached.

STOMAHESIVE® will protect skin sensitive to plaster adhesives.

STOMAHESIVE® will mould to the skin surface which may be irregular due to skin folds, scar tissue, etc.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients.

Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

Instructions for applying Stomahesive®

Intended Use: Provide protection for peristomal skin by filling skin irregularities and depressions to improve adhesion of ostomy systems.

Indications for Use: For use with a colostomy, ileostomy or urostomy.

For maximum protection it is important that Stomahesive® fits closely to the stoma. Using small scissors and a cutting guide, cut a centre hole to fit the stoma accurately.

Remove the backing paper from the Stomahesive®.

Place the Stomahesive® over the stoma and smooth down.

Hints for routine skin care when using Stomahesive®

- Clean skin thoroughly and pat dry.
- Stomahesive® should be fashioned to fit the stoma as closely as possible so that the skin around the stoma is completely protected.
- If the area around the stoma is irregular, which may permit leakage, Orabase® or Stomahesive® paste should be used as a filling before applying the Stomahesive® square.
- Small amounts of adhesive residue which may be left behind after removal need not be completely removed prior to re-application.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

© 2023 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Limited
UNOMEDICAL A/S