Rx Only / Medicamento de venta con receta / **Sur ordonnance /** RX / 經處方藥方銷售 / 經處方藥方銷售 / RX

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ENGLISH

PRODUCT DESCRIPTION

Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber is a soft, sterile, non-woven pad dressing composed of hydrocolloid fibers (sodium carboxymethylcellulose). This conformable and highly absorbent dressing absorbs wound fluid and creates a soft gel, maintains a moist environment which supports the body's healing process and aids in the removal of unnecessary material from the wound (autolytic debridement), without damaging newly formed tissue.

This primary dressing should be used with a secondary cover dressing.

INDICATIONS

Under the supervision of a healthcare professional, Aquacel Extra Hydrofiber™ Dressing with Strengthening Fiber may be used for the management of:

- Leg ulcers, pressure ulcers (Stage II-IV) and diabetic ulcers
- Surgical wounds (post-operative, donor sites, dermatological)
- Partial thickness (second degree) burns
- Traumatic or surgical wounds left to heal by secondary intention such as dehisced surgical incisions
- Surgical wounds that heal by primary intent, such as dermatological and surgical incisions (e.g., orthopedic and vascular)
- Traumatic wounds
- Wounds that are prone to bleeding, such as wounds that have been mechanically or surgically debrided and donor sites
- Painful wounds

CONTRAINDICATIONS

Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components.

PRECAUTIONS AND OBSERVATIONS

- Caution:** Sterility is guaranteed unless pouch is damaged or opened prior to use.
- This device is for single-use only and should not be re-used. Re-use may lead to increased risk infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.
- Should you observe irritation (reddening, inflammation), maceration (whitening of skin), hypergranulation (excess tissue formation) or sensitivity (allergic reaction), consult a healthcare professional.
- If you have difficulty removing the dressing, the dressing should be fully saturated with sterile saline and allowed to soak into the dressing.
- Because Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber provides a moist environment that supports the growth of new blood vessels, the delicate newly formed blood vessels may occasionally produce blood stained wound fluid.
- The dressing may be used on infected wounds only under the care of a healthcare professional.
- This wound dressing should not be used with other wound care products without first consulting a healthcare professional.

In addition, for leg ulcers, pressure ulcers, diabetic ulcers, second degree burns, donor sites, surgical or traumatic wounds left to heal by primary or secondary intention:

- Treatment of wounds listed above should be under the supervision of a healthcare professional.
- Appropriate supportive measures should be taken where indicated (e.g. use of graduated compression bandaging in the management of venous leg ulcers or pressure relief measures in the management of pressure ulcers).
- Colonization of chronic wounds is common and is not a contraindication to the use of the dressing. The dressing may be used on infected wounds under medical supervision together with appropriate therapy and frequent monitoring of the wound.
- The control of blood glucose, as well as appropriate supportive measures, should be provided with diabetic foot ulcers.
- In second degree burns, consider alternate surgical procedures if the wound has not reepithelialized after 14 days.
- Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber is not intended for use as a surgical sponge.

- The use of Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber has not been studied in wounds due to herpes simplex or impetigo.**

DIRECTIONS FOR USE

- Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.
- Apply the dressing to the wound and cover with a moisture retentive dressing (i.e., DuoDERM™ Extra Thin, Versiva™ XC™) gauze, or other appropriate dressing. See individual package inserts for complete instructions for use. Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber should overlap 1cm or ½ inch onto the skin surrounding the wound.
- All wounds should be inspected frequently. Remove the Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber when clinically indicated (i.e., leakage, excessive bleeding, suspicion ofinfection).
- The Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber is designed to remain in place for a maximum of seven days. The dressing should be changed when it is saturated with wound fluid or if the cover dressing is leaking or the cover dressing's edges are bunching or rolling up.
- In donor sites, Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber may be left in place for up to 14 days.

FOR SECOND DEGREE BURNS

- The Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber should overlap 5cm or 2 inches onto the skin surrounding the burn or other adjacent Aquacel Hydrofiber Wound Dressing.
- The Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber should be covered with sterile gauze and secured with medical tape.
- Remove the gauze cover dressing periodically and inspect the Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber while it is still in place on the burn.
- Remove the Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber when clinically indicated (e.g., leakage, excessive bleeding, clinical signs ofinfection). **For second degree burns, the Aquacel™ Extra™ Hydrofiber™ fiber Dressing with Strengthening Fiber may be left in place for up to 14 days provided there is no clinical evidence ofinfection. As the burn wound reepithelializes, the Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber will detach or be easily removed.**

FOR DRY AND MINIMALLY-EXUDING WOUNDS

- Place the Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber in the wound and then wet with sterile saline over the wound area only. The vertical absorption properties of Aquacel™ Extra™ Hydrofiber™ Dressing with Strengthening Fiber will help to maintain the moist area over the wound only and reduce the risk of maceration.
- Cover the dressing with a moisture retentive dressing such as DuoDERM™ Extra Thin to avoid drying out of the dressing and subsequent dressing adherence to the wound.

This product is for single use only and is supplied sterile. Discard any unused portion of the product after dressing the wound.

If the immediate product packaging is damaged, do not use.

Store in a cool, dry place.

If further information or guidance is needed, please contact ConvaTec's Professional Services.

Made in UK

© 2025 Convatec

™/® indicates a trademark of the Convatec Group

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Getz Bros. Philippines, Inc.
Importado por: Convatec PERU S.A.C.