VARIHESIVE

INSTRUCTIONS FOR USE

VARIHESIVE is prepared from non-irritating ingredients, (gelatin, pectin, carboxymethylcellulose and polyisobutylene), providing protection to the skin.

VARIHESIVE is a protective wafer for use in the management of ileostomies, colostomies, ileal conduits and fistulae management.

VARIHESIVE protects skin while healing takes place and provides a surface to which appliances may be attached.

VARIHESIVE has a double sided adhesive quality.

VARIHESIVE will protect skin sensitive to plaster adhesives.

VARIHESIVE will mould to the skin surface which may be irregular due to skin folds, scar tissue, etc.


Instructions for applying Varihesive

For maximum protection it is important that Varihesive fits closely to the stoma. Using small scissors and a cutting guide, cut a centre hole to fit the stoma accurately.

Remove the backing paper from the Varihesive.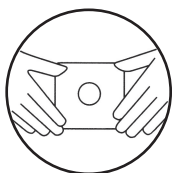

Place the Varihesive over the stoma and smooth down.

Hints for routine skin care when using Varihesive

1. Clean skin thoroughly and pat dry
2. Varihesive should be fashioned to fit the stoma as closely as possible so that the skin around the stoma is completely protected.
3. If the area around the stoma is irregular, which may permit leakage, Orabase or Stomahesive paste should be used as a filling before applying the Varihesive square.
4. Small amounts of adhesive residue which may be left behind after removal need not be completely removed prior to re-application.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

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Disposal

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

We pride ourselves on the quality of our products and the stringent testing they undergo before leaving our factory.

However, should you experience any difficulty with any of our products, please let us know.

© 2022 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Inc.
Greensboro, NC 27409
USA
1-800-422-8811

 ConvaTec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd.
1800 339 412 Australia
0800 441 763 New Zealand

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