en ENGLISH

**INDICATIONS:** For use on adhesives, dressings and appliances on skin.  
**DESCRIPTION OF DEVICE & CONTENTS:** ESENTA™ Medical adhesive remover spray sting free (50ml)

**INTENDED USE:** Removal of adhesives, dressings and appliances from skin.

**TARGET PATIENT POPULATION:** People with adhesives, dressings and appliances on skin.

**INTENDED USER:** Health Care Professionals, Caregivers and Ostomates.

**INSTRUCTION FOR USE:**

1. Spray around the edge of the adhesive and wait a few seconds.

2. Gently peel adhesive away from the skin using additional spray to ease removal if necessary.

**INFORMATION:** Dries in a few seconds. Sprays at any angle. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

**CAUTION:**

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).

- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.

- Avoid inhalation.

For external use only.

**WARNINGS:** Use only as directed. Pressurised container: may burst if heated.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children.

**EXTREMELY FLAMMABLE**  
Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

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