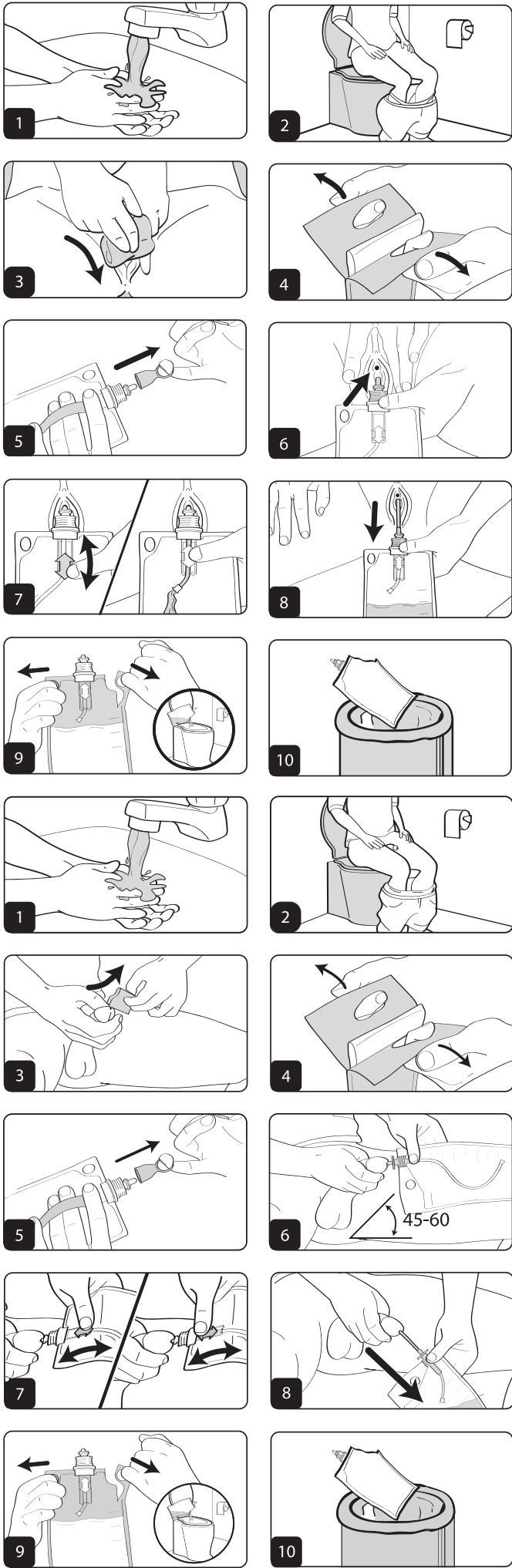
Closed system intermittent urinary catheter for male and female / Sonde/Cathéter urinaire intermittent à système fermé pour homme et femme / Catéter urinário intermitente de sistema cerrado para hombres y mujeres / Sistema fechado de cateter intermitente de homens e mulheres / 남성, 여성을 위한 밀폐형 간헐적 도뇨 카테터

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EN ENGLISH

These instructions for use must be read and understood in parallel with the pictorial instructions for use also enclosed prior to using this device. The individual steps in the pictorial instructions, showing the preparation and performance of catheterisation, must be followed. The device must only be used when prescribed by an appropriately qualified, trained healthcare professional. The patient or carer must follow the advice of the healthcare professional.

Indications for use:
The Cure Dextra® Catheter is an intermittent urinary catheter attached to a collection bag that is inserted through the urethra and indicated for the purpose of bladder drainage for males and females. The urinary catheter comes in a variety of sizes packaged sterile for single-use.

Intended Users:
Cure Dextra® Catheters are to be used by health care professionals, patients and caregivers under the direction of a health care professional.

Intended Patient Population:
Males & Females

Duration of use:
The Catheters have been designed for routine intermittent use. Devices are single-use only and must be removed immediately upon complete drainage of the bladder and disposed of. The overall usage of the device can be expected to be ongoing repeated multiple times a day.

Environment:
Professional healthcare facilities, homes, work, public bathrooms, out and about

Warnings:

- The catheter must be used immediately once removed from the pouch.
- Sterile if the package is unopened or undamaged. Do not use if package is broken.
- Check the catheter is the correct size as indicated on the packaging.
- Avoid contact with insertion tip once cap removed prior to insertion.
- Always slide the catheter slowly and smoothly when using the catheter tip advancing mechanism with gripper arrows. Never force the catheter.
- When removing the catheter, stop if more urine starts to flow. Continue again once urine stops flowing.
- The device is for single use only. Do not reuse or re-insert the device. This may lead to an increased risk of infection, transmission of viruses/bacteria, bleeding, tissue damage or pain.
- Always empty the collection bag prior to disposal.

Caution: Federal law restricts this device to sale by or on the order of a physician.

***US only**

Contact a healthcare professional:
If there is no urine flow or other usage complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge).
If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your competent authority.

Storage and Disposal
Store the product within the sealed packaging at ambient temperature.
Sterility is guaranteed unless the sealed packaging is damaged or opened prior to use. Do not use the device if the packaging is damaged or open prior to use and dispose of the device according to local regulations.

Technical Assistance
If further information or guidance is needed, visit us online at <https://curemedical.com>.
Additional paper copies of the Instructions for Use are available from Convatec Inc.

Female Intermittent Catheterization Instructions:

1. Wash hands thoroughly.
2. Position yourself comfortably. Arrange clothing out of the way.
3. Separate the labia with the thumb and forefinger to locate the urethra. Wash the urethral area from front to back. Do not wash back and forth over the urethral opening.
4. Open the Cure Dextra® Closed System package.
5. Remove the cap from the introducer tip. **DO NOT slide the catheter past the introducer tip yet.**
6. Hold the inner labia apart and insert the introducer tip into the urethra. A Support Band, on the bottom side of the collection bag, can give you added control and stability.
7. Once you have secured the introducer tip in place, hold the Cure Dextra®. Slide the Gripper Arrow toward your body with a full stroke so the catheter advances through the introducer tip into the urethra. Once the slider is fully forward, push the Gripper Arrow fully back. The catheter will not retract on the back stroke. Repeat full forward and back motions to advance the catheter. Continue to advance the catheter through the urethra until the catheter reaches the bladder and urine begins to flow. Keep the bottom of the urine bag down to help urine flow.
8. When urine stops flowing, slowly withdraw the catheter. If urine starts to flow again when withdrawing the catheter, stop each time it flows and let the urine drain. Then continue to slowly withdraw the catheter until the bladder is empty, and the catheter is removed.
9. To empty the bag, hold it upright and put your fingers in the openings provided at the top of the bag. Pull the finger openings apart to open the drain. Pour out the urine through the opening into the toilet.
10. Dispose of the bag properly.

Male Intermittent Catheterization Instructions:

1. Wash hands thoroughly.
2. Get in a comfortable position. Arrange clothing out of the way.
3. Hold the penis upright on the sides so that the urethra is not pinched. Retract the foreskin if the penis is uncircumcised. Wash the glans penis from the urethral opening (tip) to the base of the glans. Wash with a circular motion from the tip outward. **DO NOT wash back and forth over the urethral opening.**
4. Open the Cure Dextra® Closed System.
5. Remove the cap from the introducer tip. **DO NOT advance the catheter past the introducer tip yet.**
6. Hold the penis at a 45 to 60-degree angle from the body. Insert the introducer tip into the urethra. A Support Band, on the bottom side of the collection bag, can give you added control and stability.
7. Slide the Gripper Arrow toward your body with a full stroke so the catheter advances through the introducer tip into the urethra. Once the slider is fully forward, push the Gripper Arrow fully back. The catheter will not retract on the back stroke. Repeat full forward and back motions to advance the catheter. Using full strokes, gently advance the catheter until you meet resistance at the urethral sphincters. Do not pull the catheter in and out when you meet resistance; just apply gentle but firm continuous pressure until the catheter advances into the bladder. Continue to advance the catheter through the urethra until the catheter reaches the bladder and urine begins to flow. Keep the bottom of the urine bag down to help urine flow.
8. When urine stops flowing, slowly withdraw the catheter. If urine starts to flow again when withdrawing the catheter, stop each time it flows and let the urine drain. Then continue to slowly withdraw the catheter until the bladder is empty, and the catheter is removed. If you are uncircumcised, make sure you bring the foreskin back over the glans penis when you are finished.
9. To empty the bag, hold it upright and put your fingers in the openings provided at the top of the bag. Pull the finger openings apart to open the drain. Pour out the urine through the opening into the toilet.
10. Dispose of the bag properly.

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Convatec Inc. 7815 National Service Road, Suite 600 Greensboro, NC 27409 USA 1-800-422-8811