eakin Wound Pouches
Bolsas Eakin para heridas
Sacs pour plaies Eakin

EN

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

eakin Wound Pouches are flexible pouches designed to offer skin protection and contain drainage from wounds, fistulae and difficult stoma sites.

INDICATIONS

eakin Wound Pouches can be used to dress and drain wounds, fistulae and irregular or multiple stomas.

CONTRAINDICATION

eakin Wound Pouches should not be used on individuals who are sensitive to or who have developed a sensitivity or allergic reaction to the product.

CHOOSING THE CORRECT POUCH

Measure the length and width of the site to be dressed, and choose the appropriate pouch.

APPLICATION

1. Draw the shape of the site on the template provided and cut along the trace line.
2. Align the template with the skin barrier on the pouch and trace the desired shape of the site.
3. Before cutting the skin barrier along the line marking the shape of the site, allow a small amount of air to enter the pouch by

EN

gently spreading the pouch walls. Cut the desired shape taking care not to puncture the opposite wall of the pouch with the scissors.

4. Prepare the patient by cleaning the skin around the site with water and pat dry. Make sure the skin remains clean, dry, and free from any solvent or greasy substances. The eakin **Cohesive®** on the face piece of the pouch will adhere to moist skin; however, a better result is obtained when the skin is dry.
5. Remove the release paper from the skin barrier and position the skin barrier around the site to be managed.
6. Apply gentle pressure around the skin barrier for at least 30 seconds to ensure adherence.

DRAINING THE POUCH

The pouch may be opened by holding the bottom of the outlet pocket and gently releasing the outlet. The outlet may be closed and repositioned as follows:

1. Open the outlet.
2. Fold the outlet end extending it above the pocket.
3. Fold the doubled outlet level with the bottom pocket.
4. Tuck the folded outlet into the pocket to secure the closure.

Empty the pouch at regular intervals. Frequent emptying of the pouch prevents 'drag' and prolongs its useful life.

All eakin Wound Pouches are also available with a 8.5mm diameter bung closure attachment for night drainage if required.

EN

A separate access window (839280) is available which can be placed anywhere on the larger pouches to gain access.

REMOVAL

1. Gently peel the skin barrier from the body.
2. Residue can be removed from the skin by rolling and peeling.
3. An emulsifying agent or medical adhesive remover can be used topically to remove tackiness from either the body or the skin barrier.

WARNING: If a Medical Adhesive Remover is used some solvents are highly flammable. All should be used in a well-ventilated area and not in the presence of a flame.

DISPOSAL

Adhere to local disposal requirements/regulations.

STORAGE CONDITIONS

Storage temperature

Keep dry. Avoid high humidity.

LOT Lot/control number

Note: Should irritation occur, discontinue use and consult a healthcare professional.

For more information on Convatec products, contact us at the address listed.

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©2024 TG Eakin Ltd.
Convatec, Inc.
Convatec Canada Ltd.