Stomahesive™ Seal

Instructions for Use / Instrucciones de uso / Instructions d'utilisation / Instruções de utilização / 使用説明 / ご使用上の注意 / Gebrauchsweisung lessen / Istruzioni per l'uso / Gebruiksaanwijzing / Brugsanvisning / Käyttöohjeet / Brugsanvisning / Bruksanvisning / Instrukcja użtykowania / Návod k použití / Návod na použitie / Kasutusjuhend / Üputé za uporabu / Kullanım Talimatı / تطبيقات استخدام / Инструкции за употреба / Használati utasítás / Naudojimo instrukcija / Lietosanás pamācība / Instrucțiuni de utilizare / Navodila za uporabo

ENGLISH

Product Description: Stomahesive™ Seals are moldable, moisture-absorbing skin barriers designed to help protect the skin from contact with body fluids.

Intended use: Provide protection for peristomal skin by filling skin irregularities and depressions to improve adhesion of ostomy systems.

Indications for use: For use with a colostomy, ileostomy or urostomy.

Indications: Stomahesive™ Seals can be used for skin protection as:

- an ostomy seal for ileostomies, colostomies and urostomies
- an external packing agent on uneven skin

Contraindications:

Do not use to cover broken skin or open wounds, or by individuals who are sensitive to or who have developed sensitivity or allergic reaction to the product.

Precautions and Observations:

• Caution: Patients with a flush urostomy when associated with megaeumeters and the absence of a urinary reservoir, should know that there have been reported cases of adhesives found in the urinary tract. Initiate appropriate clinical management and evaluate the continued use of the adhesive product if a mass is appreciated.

• Care should be taken when applying the seal and the ostomy device to protect the stoma and the surrounding skin from trauma or injury.

Application:

1. Make sure the skin is clean and dry.

2. Remove the release liner from both sides of the seal.

3. Gently roll, roll, stretch or compress the seal to fit around the stoma. The seal may be cut, stacked or repositioned to fit around difficult sites.

4. Gently apply and smooth the seal towards the base of your stoma. While applying a little pressure, hold the seal in place for about 30 seconds.

5. Apply the Ostomy appliance directly on to the surface of the seal and hold in place for 30 seconds with gentle pressure until secure.

6. **Wear as long as comfortable and secure. Wear time may vary.**

Note: Should irritation occur, discontinue use and consult a healthcare professional.

Removal:

Gently remove the pouching system from the body. Any residue can be removed from the skin by rolling and peeling or by using ESENTA™ String Free Adhesive Remover.

Disposal: This device is for single-use only and should not be reused. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

© 2023 Convatec Inc.

*™ indicates trademarks of Convatec Inc.

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