One-Piece Urostomy Pouch with Stomahesive®† Skin Barrier

Active Life Urostomy Pouches are one-piece appliances consisting of a Stomahesive disc. Specifically designed for people who need the protection of the Stomahesive Skin Barrier, but who prefer a one-piece appliance, they offer the following advantages:

- The unrivalled protection of the Stomahesive Skin Barrier.
- A soft, quiet odour-proof film.
- Pre-cut sizes for ease of application, or a starter hole (19mm) for those people who prefer to tailor the appliance to fit the stoma.
- The choice of a belt fitting if required.

CHOOSING THE CORRECT HOLE SIZE:

Pre-cut holes are available in the following sizes:

- 25mm (1")
- 32mm (1 1/4")
- 38mm (1 1/2")
- 45mm (1 3/4")

The diameter of the stoma should be measured and a hole size selected which is the same size or a little larger than the stoma.

The Starter hole is 19mm in diameter and can be cut to the same size and shape as the stoma, up to a maximum diameter of 45mm (1 3/4").

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

GENERAL INSTRUCTIONS

- Clean the skin around the stoma with soap and warm water and pat dry.
- When using **Stomahesive*** products, barrier creams and solutions are best avoided as they impair good adhesion.
- If the surface of the skin around the stoma is uneven, depressions or creases may be filled with a filler, such as **Stomahesive Paste**. Refer to diagram ❶.
- **Active Life Pouches** need not be removed while bathing.
- Small amounts of adhesive residue which may remain on the skin when changing the pouch need not be completely removed.

PREPARING THE ACTIVE LIFE UROSTOMY POUCH FOR APPLICATION

Pre-cut holes

- Measure the maximum diameter of the stoma and select a hole size which is the same size or a little larger than the base of the stoma. Before positioning the pouch over the stoma check that there will be no gaps of exposed skin between the edge of the pre-cut hole and the base of the stoma.

1. If there are likely to be exposed areas of skin, a filler such as Stomahesive* Paste can be used around the base of the stoma to protect the skin, prior to the application of the pouch.

Starter hole

2. Using the guidelines on the release paper on the back of the Stomahesive disc, trace the outline of the stoma.
3. Before cutting the Stomahesive* disc, separate the front and back panels so as not to cut the front panel of the pouch.
4. Using a small pair of scissors, preferably curved, enlarge the starter hole to match the traced outline. **Do not cut beyond** the last circle on the guide.

APPLYING THE ACTIVE LIFE UROSTOMY POUCH

- Remove the single release paper from the skin barrier.
 - Hold the appliance by the two edges and position the skin barrier with the hole over the stoma.
 - Press the Stomahesive* disc against the skin paying particular attention to the area closest to the stoma. During application, maintain gentle pressure on the disc to improve adhesion.
5. Fitting a belt - if a belt is required for additional support or security at night, attach the belt to the lugs provided and adjust to a comfortable fit.

USING THE TAP ON THE UROSTOMY POUCH

The pouches are provided with the tap in the open position.

6. To close the tap, bend the outlet spout (A) to meet the plug (B).
7. Press the plug firmly into the spout for complete security.
8. The plug is more easily removed for emptying if the outlet tube is squeezed immediately behind the plug.

Note: The outlet spout is designed for the attachment of a night drainage bag if required.

REMOVING THE POUCH

9. First drain the pouch. Starting at the top, gently pull away from the skin. Apply light pressure on the skin with your free hand as you peel the pouch downwards.

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© 2025 Convatec Inc.
®/™ indicates trademarks of Convatec Inc.
®/™ indicates trademarks of ConvaTec Inc.
ConvaTec Inc.
ConvaTec Limited
© 2025 ConvaTec Inc.